


9

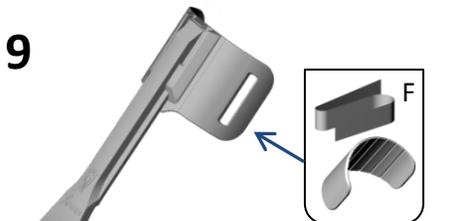


INSTALL CALF PAD & STRAP KIT  
(INSTRUCTIONS IN THE BAG)

BRACE WIDTH CAN BE  
ADJUSTED WIDER OR  
NARROWER FOR BETTER LOOK  
& FIT ON PATIENT SHOE.

(SEE THERMOFORMING GUIDELINE IF REQUIRED)

10

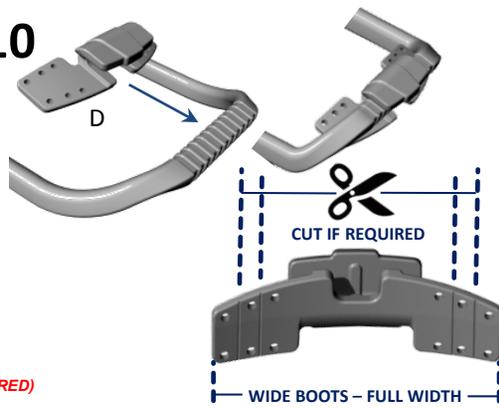

CUT IF REQUIRED

WIDE BOOTS - FULL WIDTH

11 - EASY FITTING SHOES FOR FIRST TRIAL – SNEAKERS – TENNIS SHOES



WE RECOMMEND STABLE SHOES WITH FIRM  
HEELCAP, ROBUST FABRIC AND FITTED WITH RIDGE  
ON SOLE CONTOUR TO HOLD THE BRACE IN PLACE  
AVOID MINIMALIST SHOES WITH VERY LIGHT  
FABRIC AND THIN SOLE.



14

LOOK FOR EYELETS POSITION.



15

ASK PATIENT TO DON THE SHOE  
& SELECT WIDTH FOR HOLES.



16

INSTALL TIE-WRAPS IN EYELETS.  
NEVER ATTACH AROUND THE LACES!



17

LOCK TIE-WRAPS IN REAR  
CORNERS OR BEST FIT POSITION.




18

TIGHTEN TIE-WRAPS AT MAXIMUM  
VERY STRONG FIXATION IS IMPORTANT!  
PULL FIRMLY ON LACE CLIP AND VALIDATE IT'S  
SOLIDLY ANCHORED AND BONDED TO THE SHOE.



19

HOOK THE BRACE ON THE SHOE  
AND YOU'RE READY FOR ACTION!  
SEE LACE CLIP FITTING LEAFLET  
FOR MORE FOOTWEAR OPTIONS.  
(VIDEOS TUTORIALS AVAILABLE  
ON YOUTUBE & OUR WEBSITE)

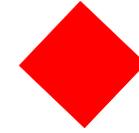


ASSEMBLY INSTRUCTIONS  
XTERN SEMI-ASSEMBLED ORTHOTIC KIT  
SIZE PEDIATRIC (100100-SA-P)



6 20553 28091 1

SHOE SIZE 9.5 US-Child -> 4.0 US Youth

2019-01-11 – 100100-SA-P XTERN

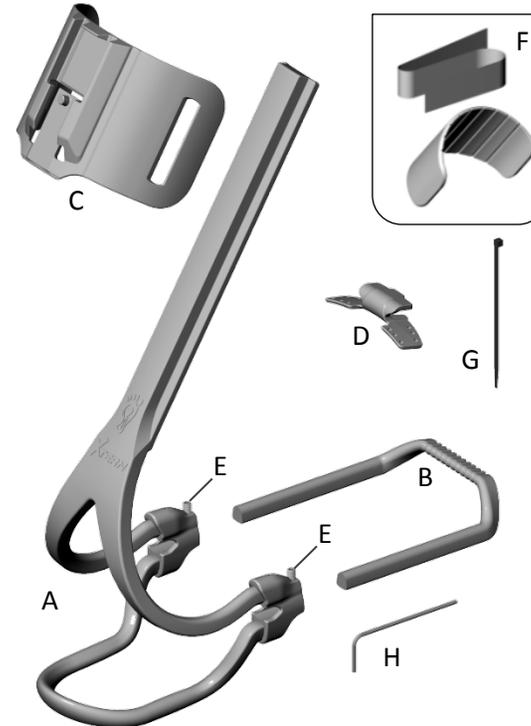

SUGGESTED TOOLS

- HEX ALLEN KEY 5/64" (INCLUDED)
- METAL HAND SAW
- MEASURING CALIPER 300MM OR MEASURING TAPE
- BENCH VISE WITH SOFT PAD JAW (PLASTIC / LEATHER)
- TRAUTMAN SANDER AND CARVER
- CUTTER PLIER
- PROPANE BLOW TORCH (IF THERMOFORMING REQUIRED)


PROTECTIVE SAFETY GEAR:

SAFETY GOGGLES, GLOVES, DUST MASK  
& ALL OTHER PROTECTION REQUIRED BY  
YOUR HEALTH & SAFETY WORK POLICY



PARTS LIST - XTERN ORTHOTIC KIT, SIZE PEDIATRIC (100100-SA-P)			
Item	Qty	PRODUCT CODE	DESCRIPTION
A	1	200000-BA0-P	MAIN BRACE BODY ASSEMBLY, SIZE PEDIATRIC
B	2	200000-U00-P	FRONT SHOE SECTION «U» SHAPE, SIZE PEDIATRIC
C	1	200000-C00-P	PLASTIC CALF SECTION, SIZE PEDIATRIC
D	5	200000-L00-P	LACE CLIP ATTACHMENT, SIZE PEDIATRIC
E	2	700000-SET-004	SET SCREW 8-32 x 5/16" 18-8 STAINLESS STEEL (ON BRACE)
F	1	500301-K00-P	COMPLETE STRAP & CUSHIONS KIT
G	20	700000-ZZZ-001	TIE-WRAP BLACK 18 LBS/4"LONG
H	1	700000-HEX-003	HEX-L-KEY 5/64" (FOR 8-32 SET SCREW)

XTERN PEDIATRIC KIT - LENGTH ADJUSTMENT CUTTING CHART															
SHOE SIZE	US-Kids	9.5C	10C	10.5C	11C	12C	12.5C	13C	1Y	1.5Y	2Y	2.5Y	3Y	3.5Y	4Y
CUTTING REF.	X	12	14	18	21	25	28	33	37	40	42	45	48	53	56

SEE GRADUATIONS ENCARVED UNDER FRONT "U" SECTION

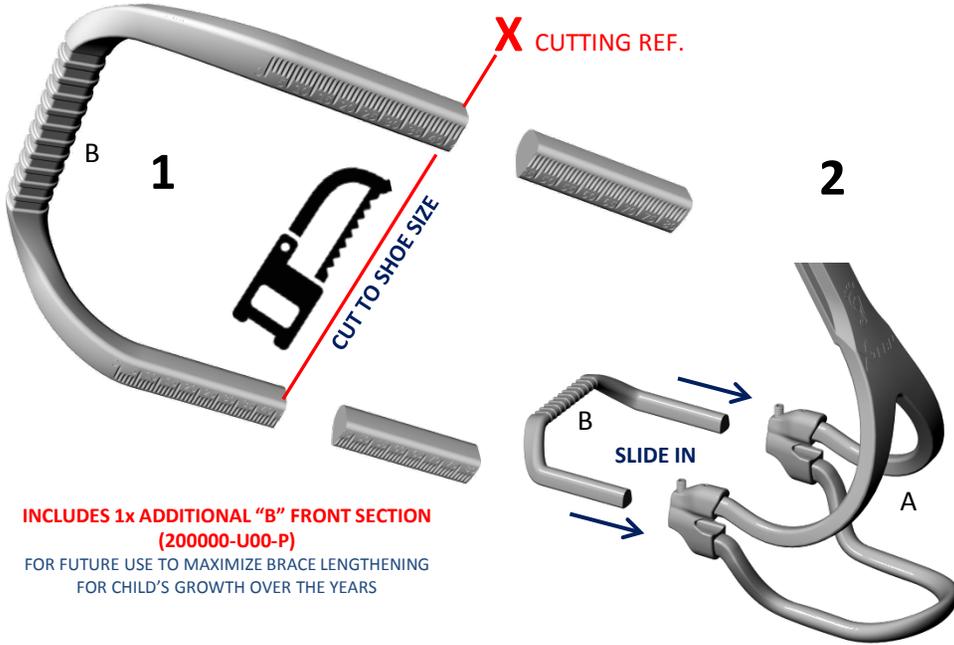

XTERN PEDIATRIC KIT - LENGTH MEASUREMENT CHART															
SHOE SIZE	US-Kids	9.5C	10C	10.5C	11C	12C	12.5C	13C	1Y	1.5Y	2Y	2.5Y	3Y	3.5Y	4Y
FINAL LENGTH	L (mm)	127	129	134	137	141	145	149	153	156	158	160	163	168	171

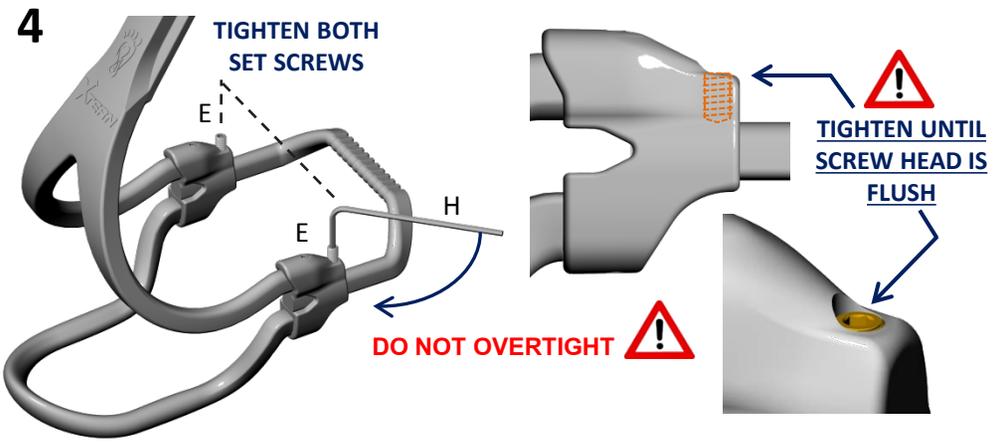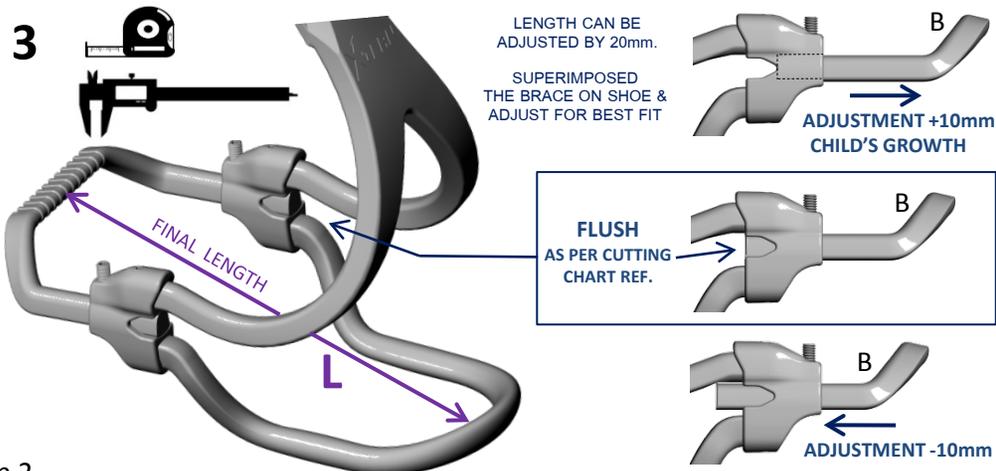

XTERN PEDIATRIC KIT - CALF SUPPORT ADJUSTMENT CHART															
SHOE SIZE	US-Kids	9.5C	10C	10.5C	11C	12C	12.5C	13C	1Y	1.5Y	2Y	2.5Y	3Y	3.5Y	4Y
CUTTING REF.	H (mm)	4	4	3	3	3	2	2	2	2	1	1	1	1	FULL

SEE NUMBERS MARKS ON PART

