turбomedorthotics.com

turbomedusa.com

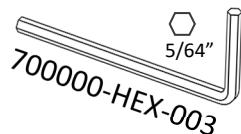
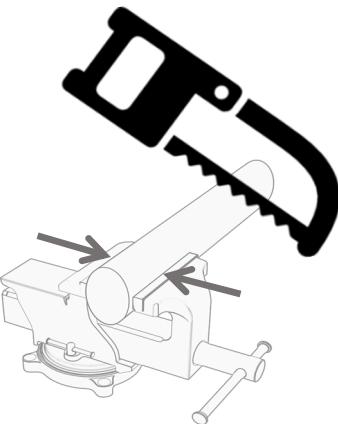
info@turbomedorthotics.com

TurboMed Orthotics Inc.
240 Sydney (101),
St-Augustin-de-Desmaures , QC
G3A 0P2 CANADA T: 1-888-778-8726

U.S. Patent No. : US 8,529,484, USD 804,044, USD 875,960, US 11,471,313
Canadian Patent & Industrial design No. : CA 2 692 534, CA 2 900 058, CA 168 202
European Patent & Community design No. : EP 2 533 734, 003075654-01/02,
GB 90030756540001, 0002, 008883326-0001, GB 6197836
Patents pending worldwide: PCT/CA2018/051062, PCT/CA2022/051826

WARNING!

Wear appropriate personal protective equipment, depending on the nature of the tool and the associated hazards. Keeping tools clean and free from grease, oil, or other substances that may affect grip or performance. Applying the appropriate force and using proper techniques while using hand tools to prevent slips, loss of control, accidental cuts, or injuries. You may use fixtures to hold the workpiece securely instead of relying on handholding. Following guidelines for safe operation, including positioning hands and fingers away from cutting edges or moving parts, and avoiding awkward or unstable positions while operating the tool. Keeping tools out of reach of children or unauthorized individuals. Maintaining a clean and well-organized work area to reduce the risk of accidents or tripping hazards. Regularly inspecting tools for damage, wear, or defects, and removing damaged tools from use until repaired or replaced.

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



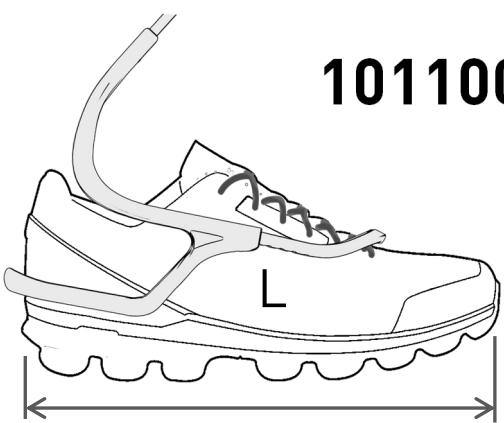
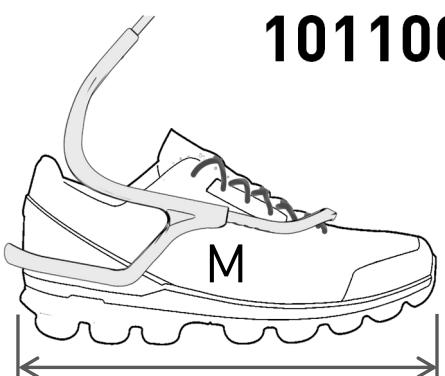
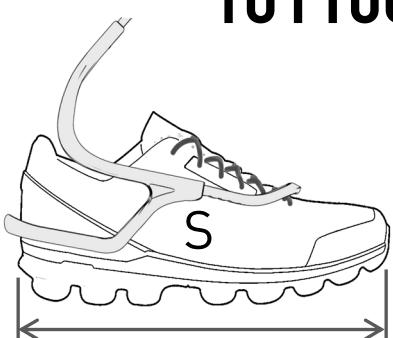
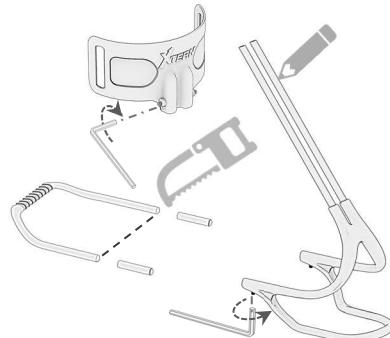
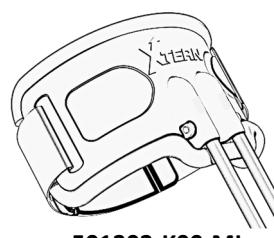
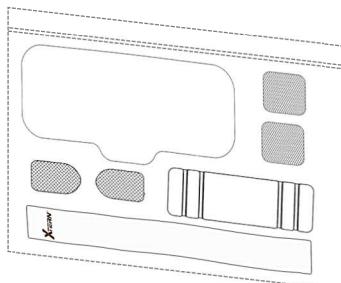
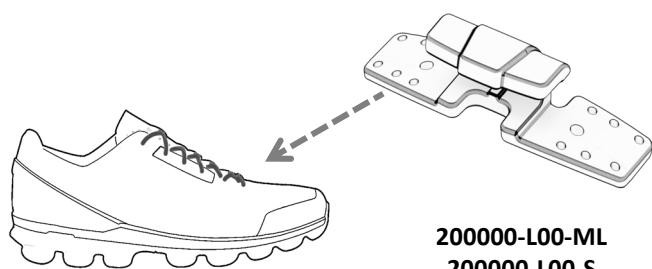
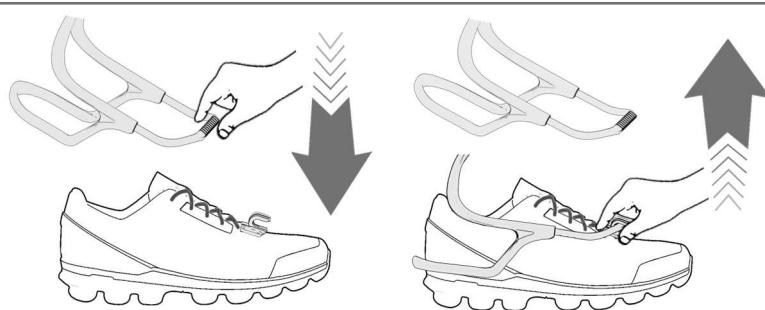
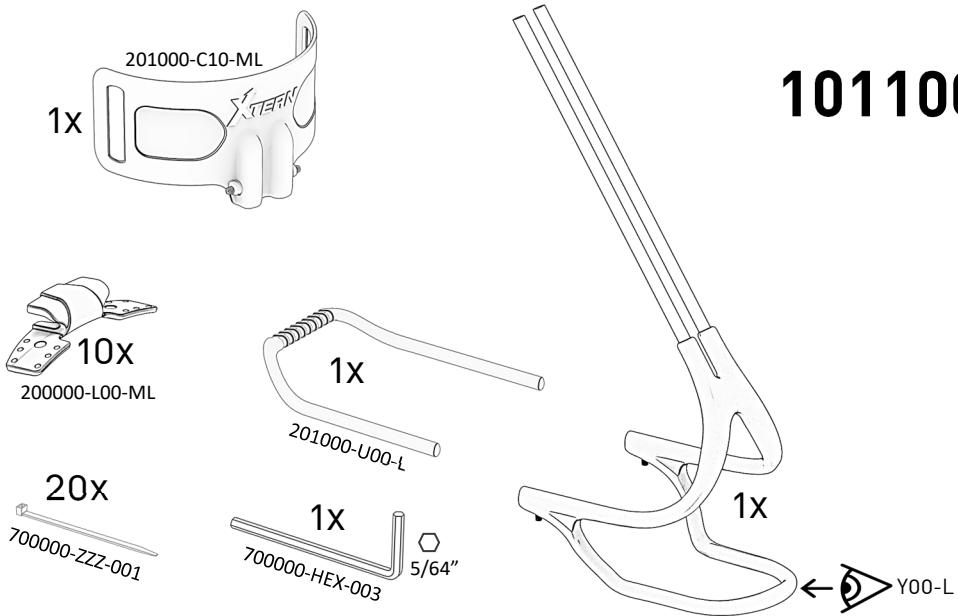
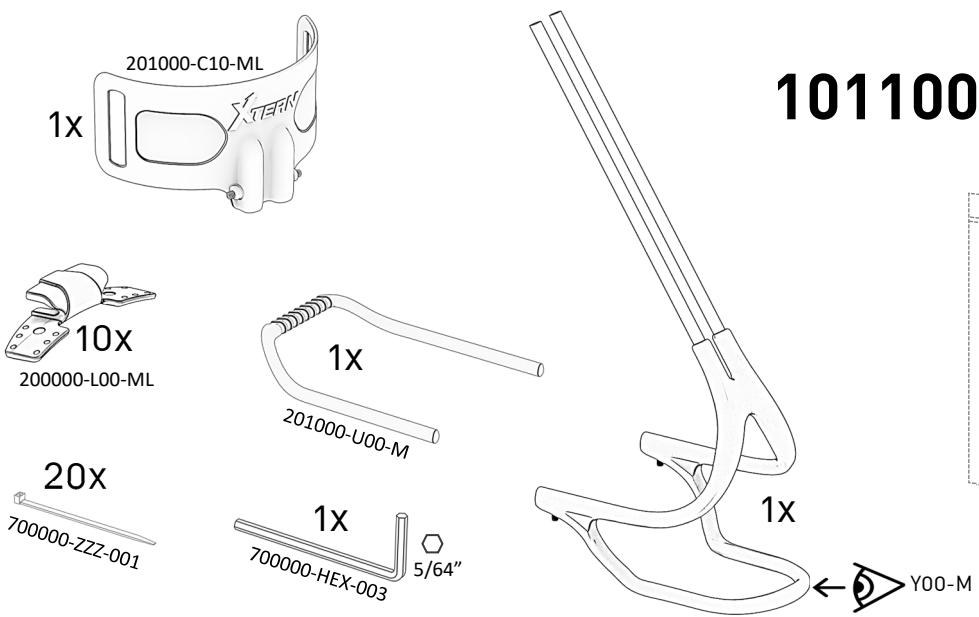
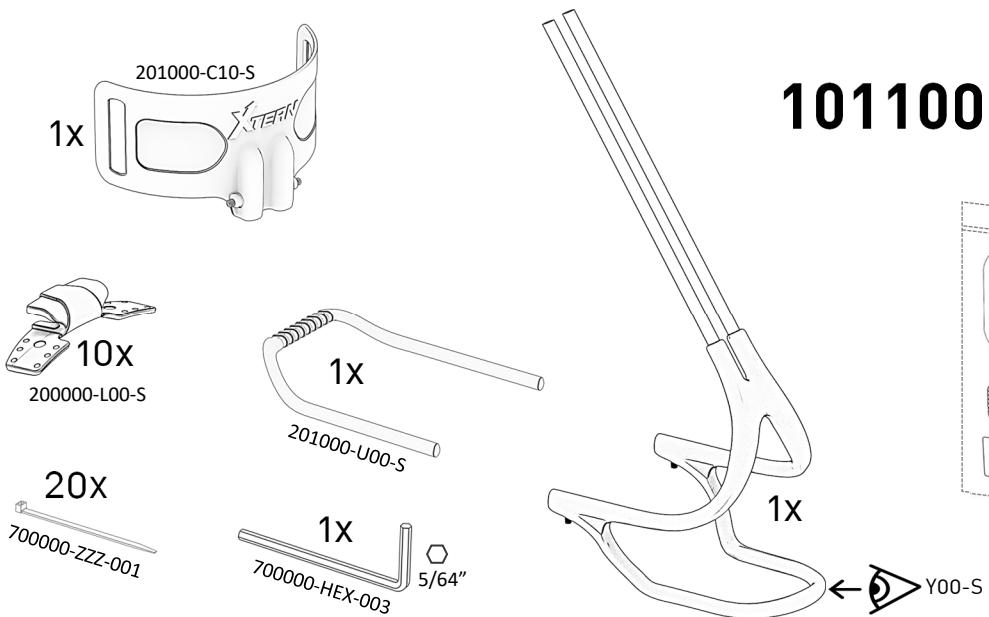
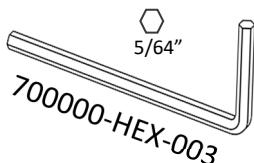
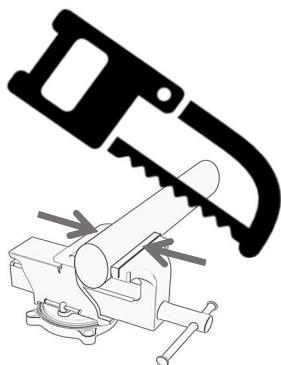
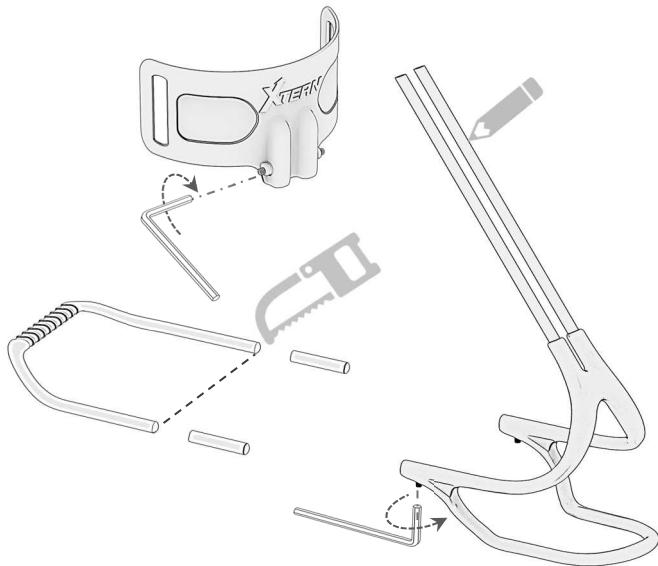
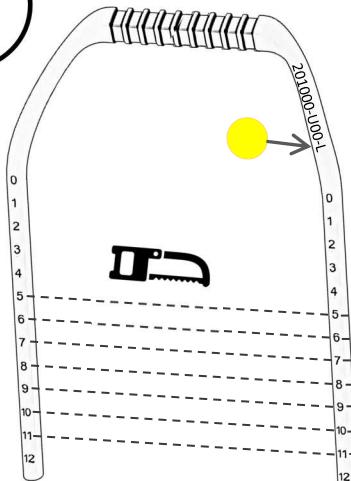
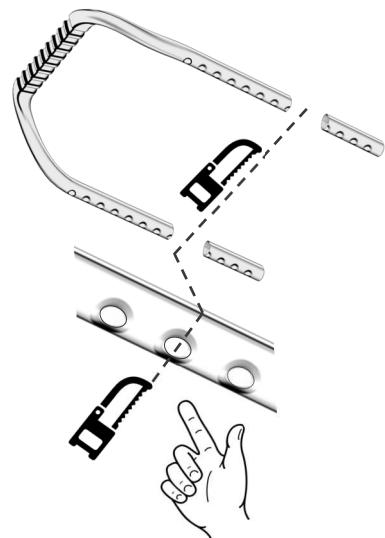
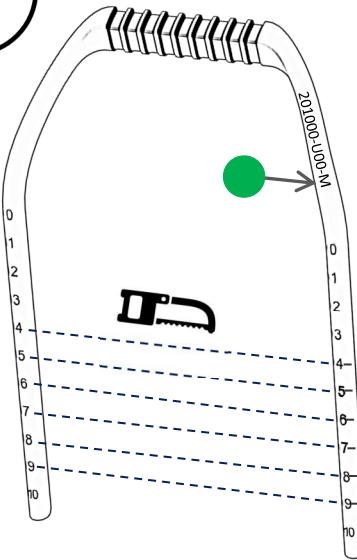
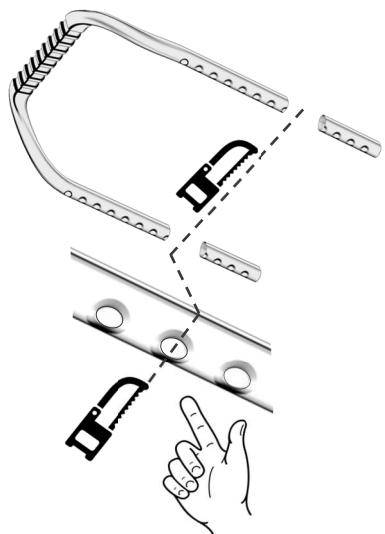
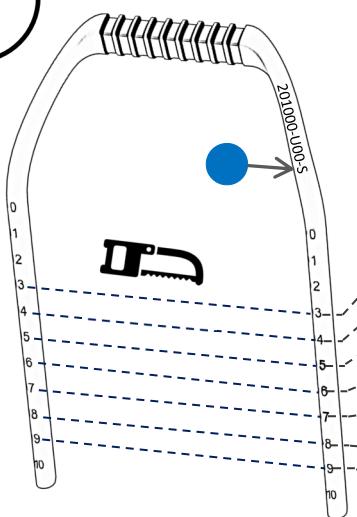
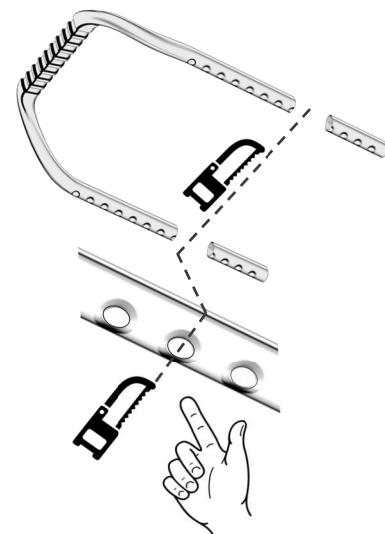
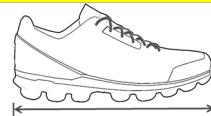
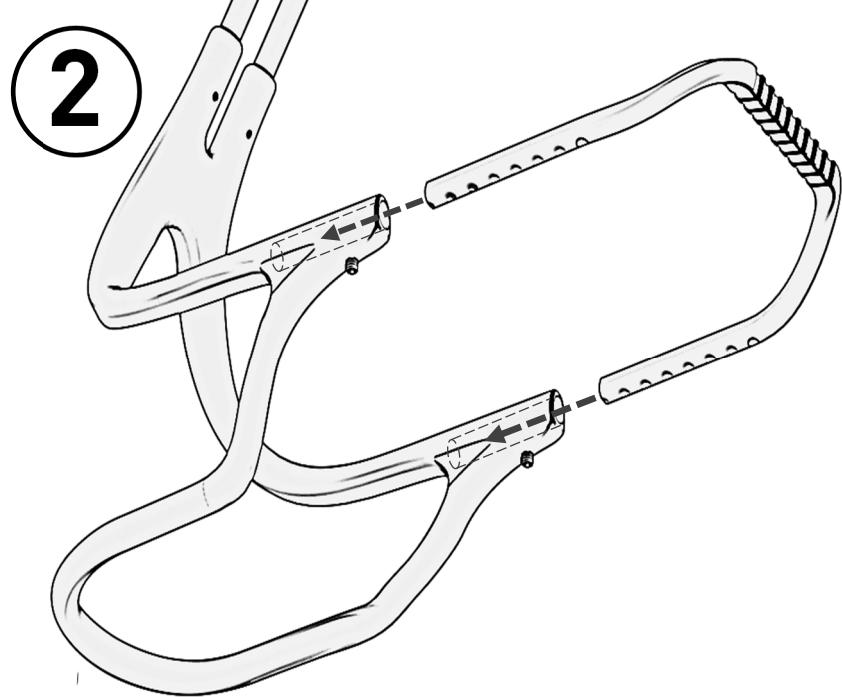
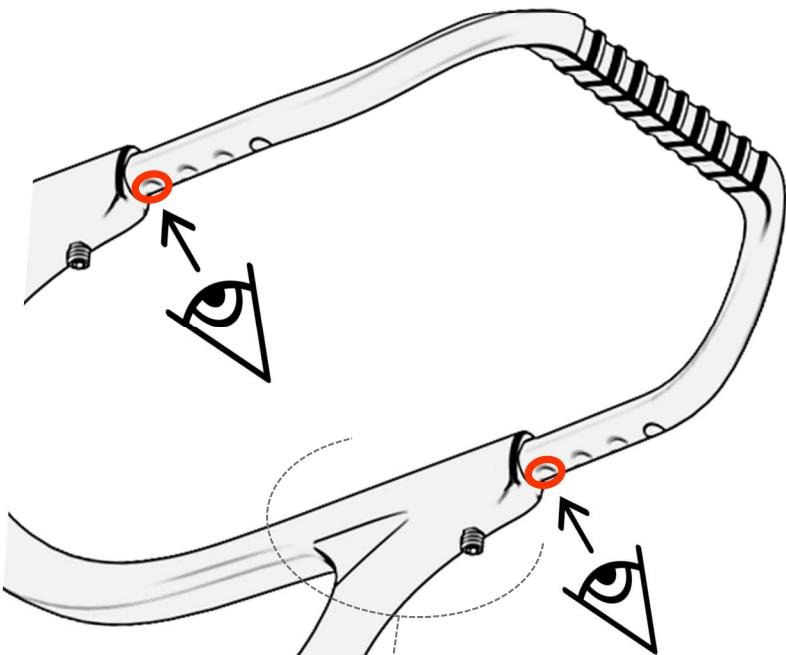
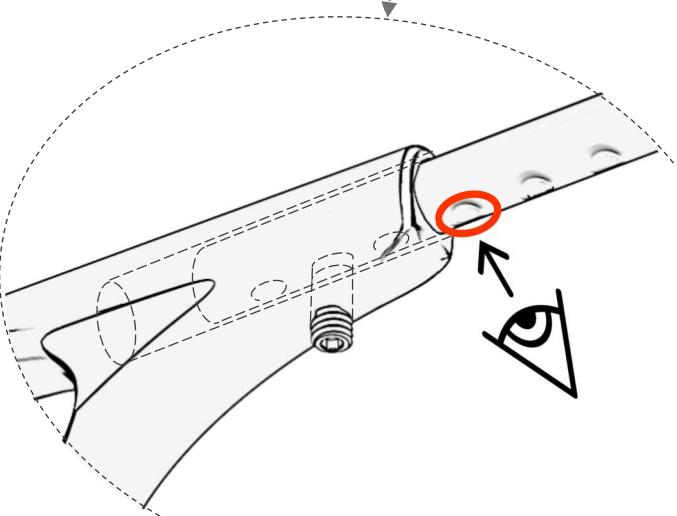
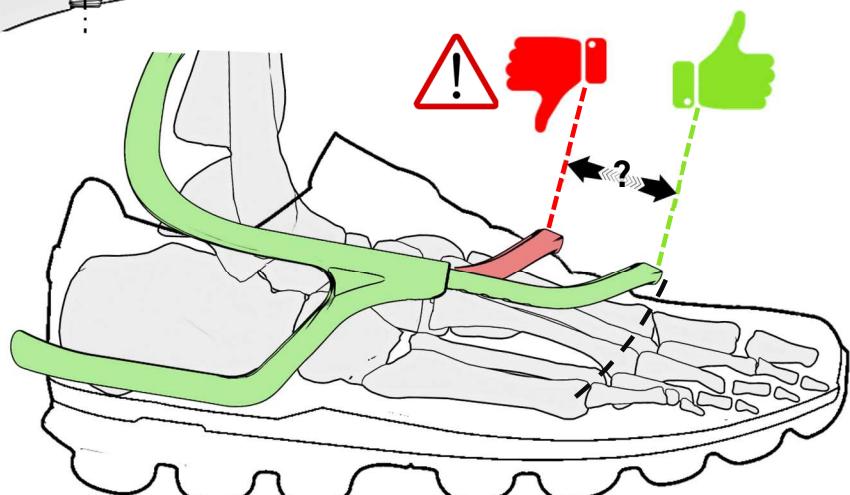
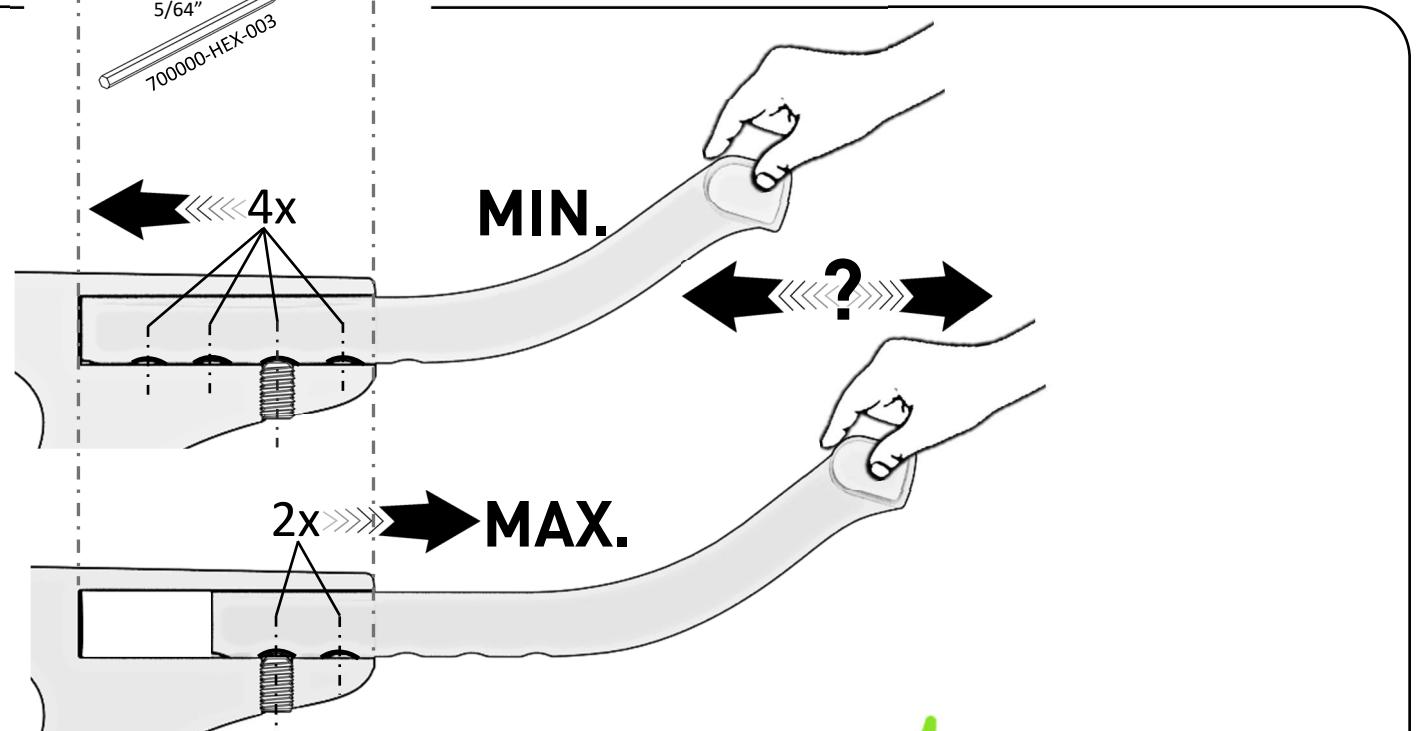
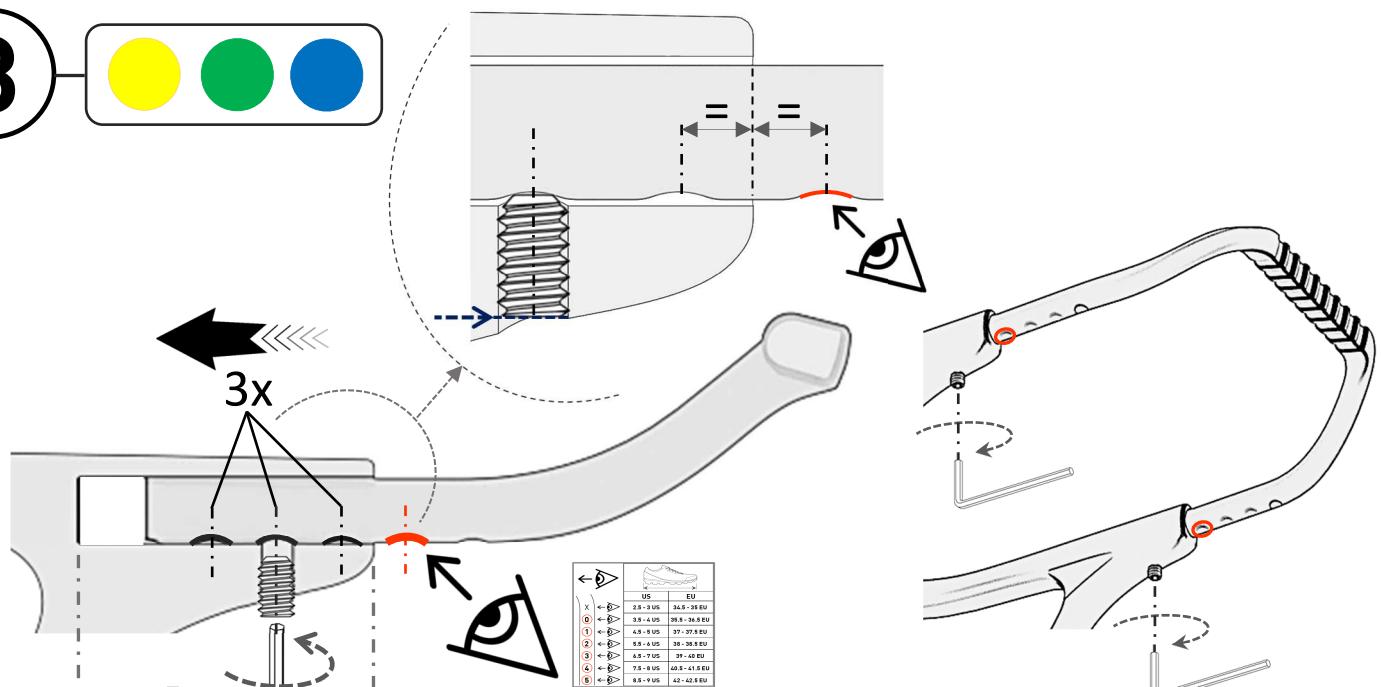
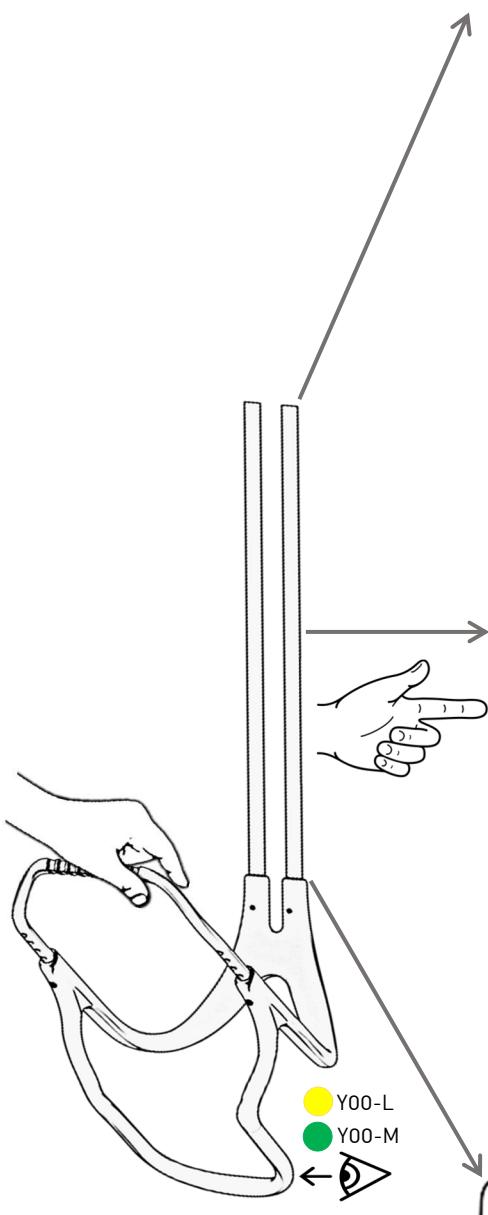
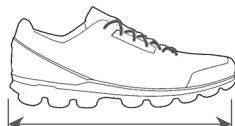
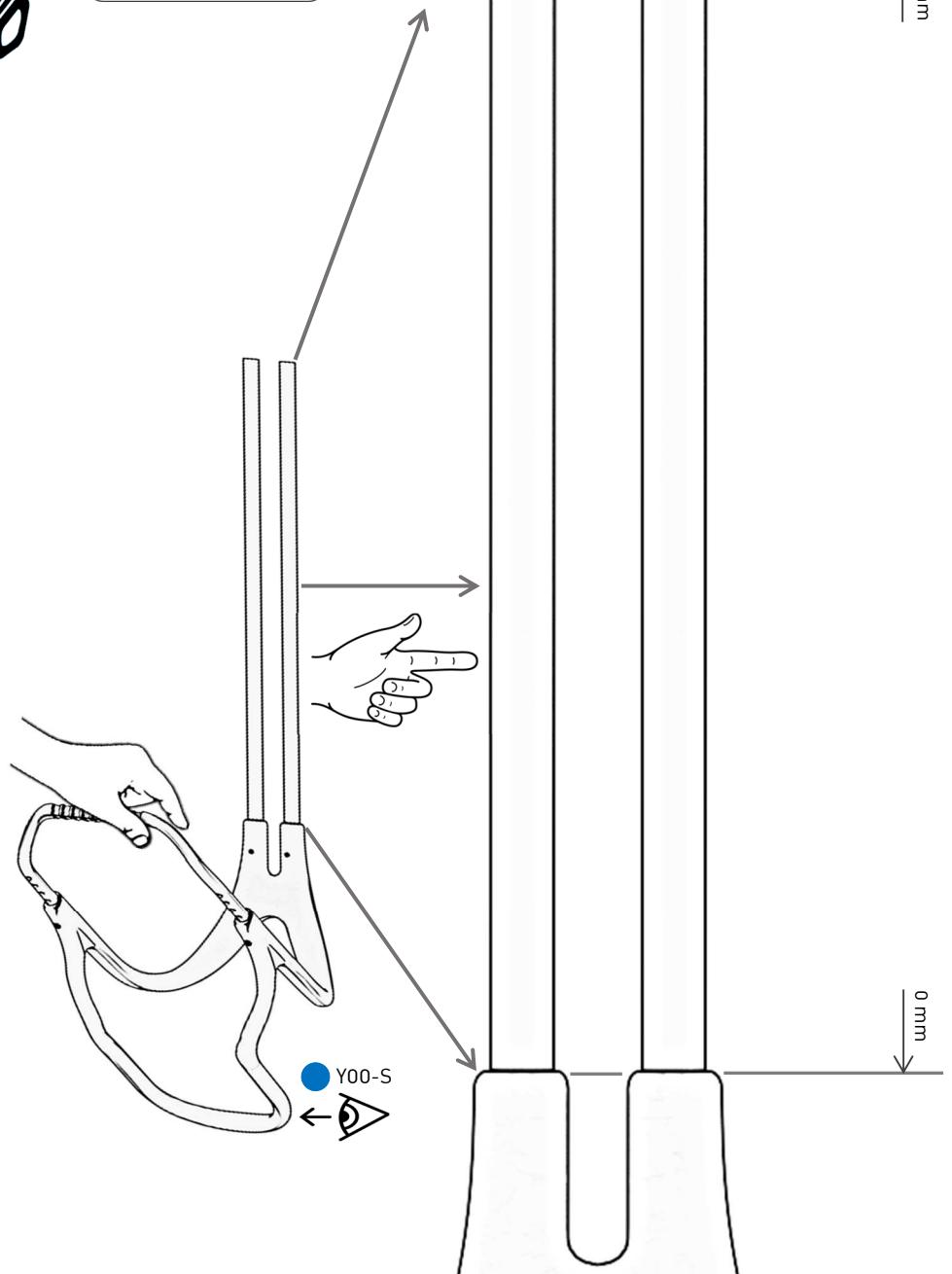
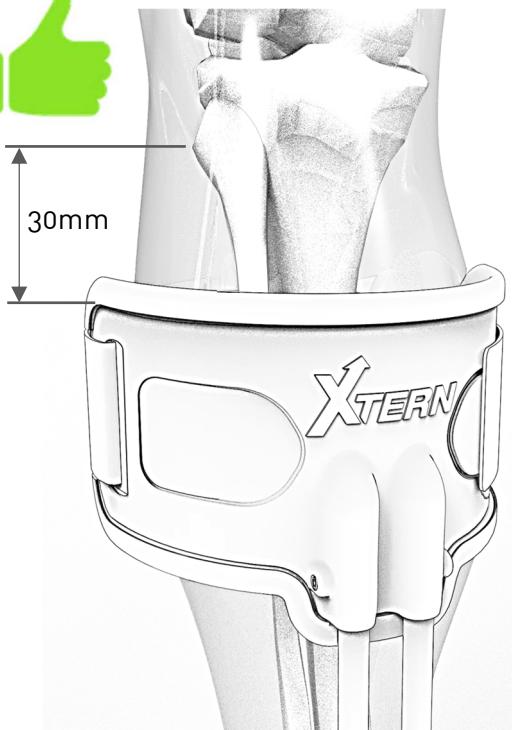
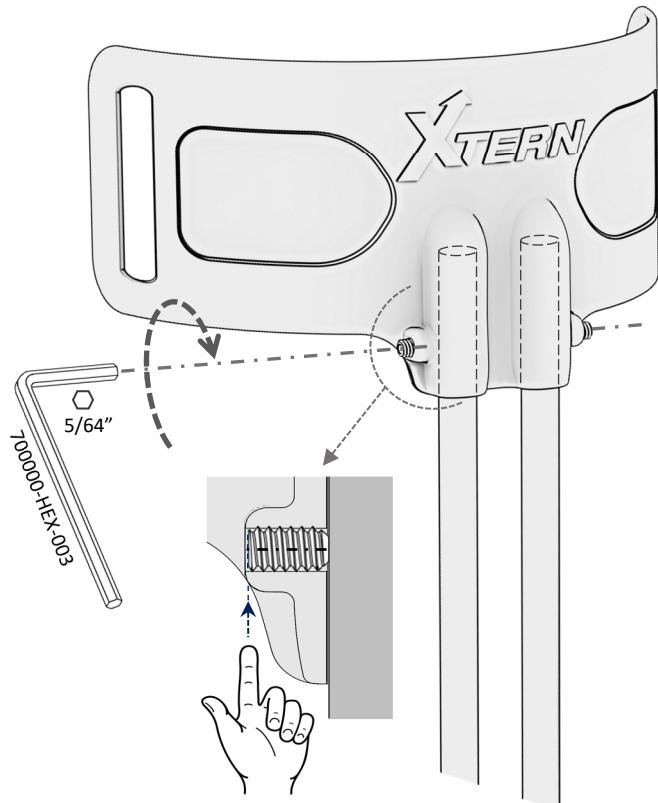
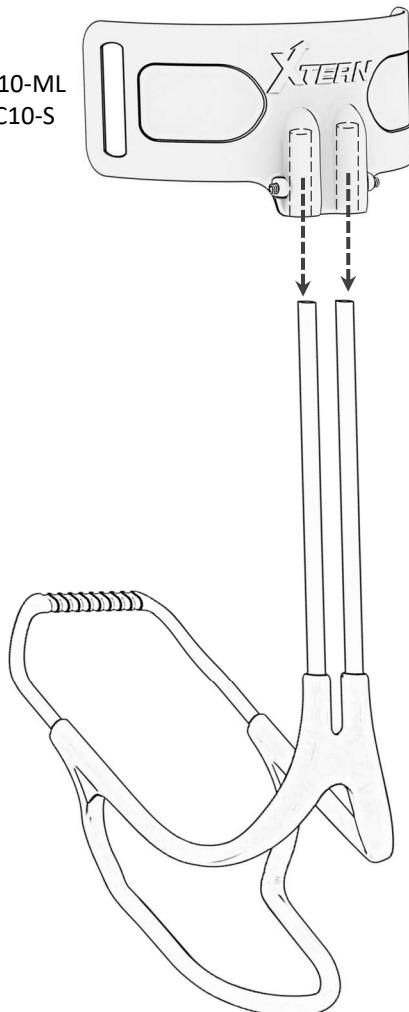
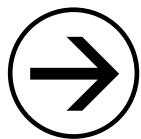
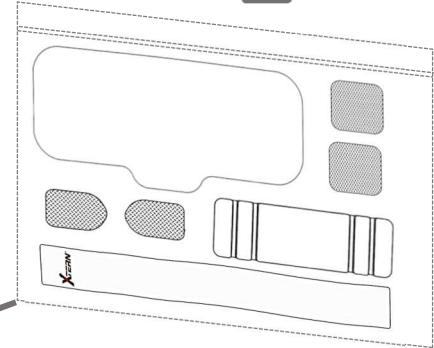
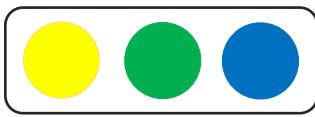
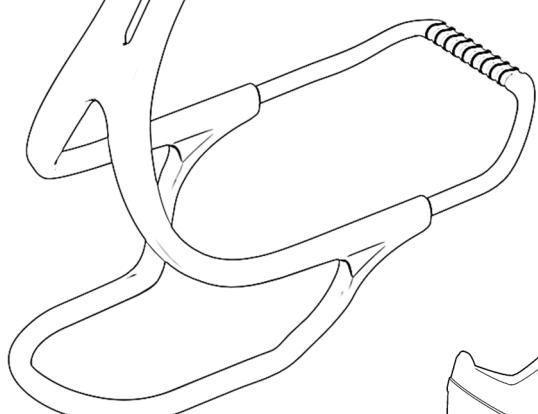
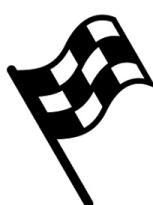
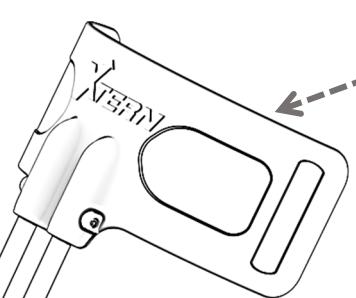
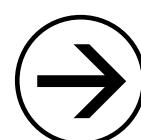
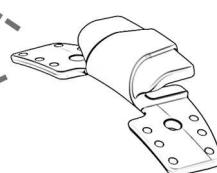
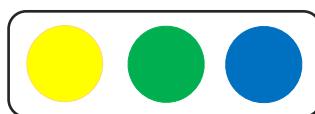
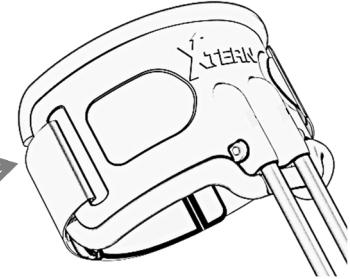
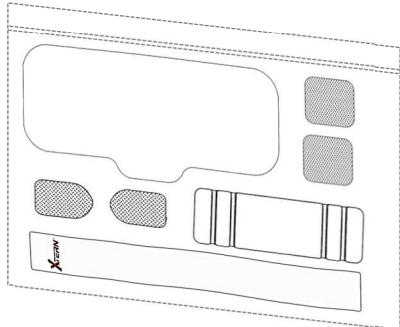
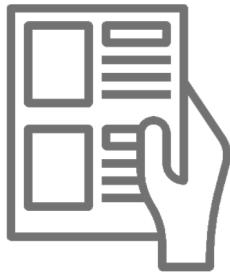
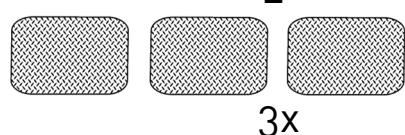
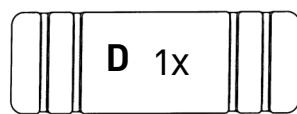
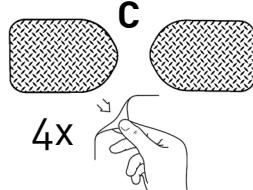
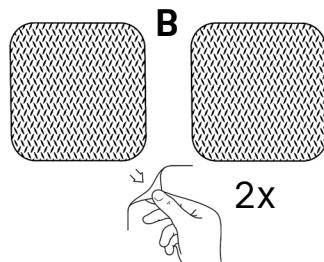
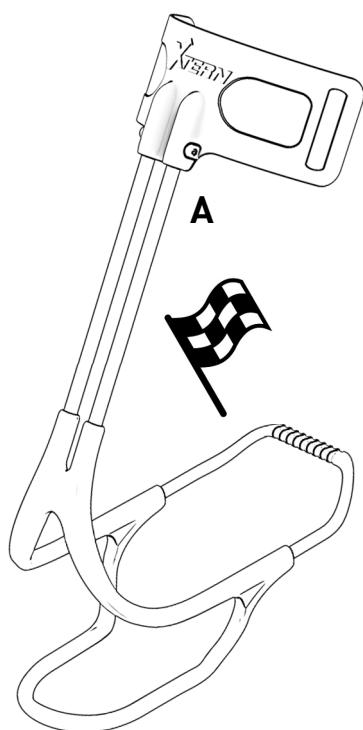
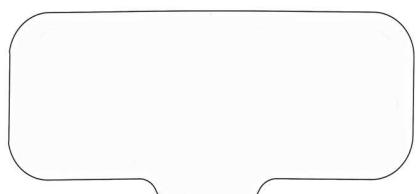
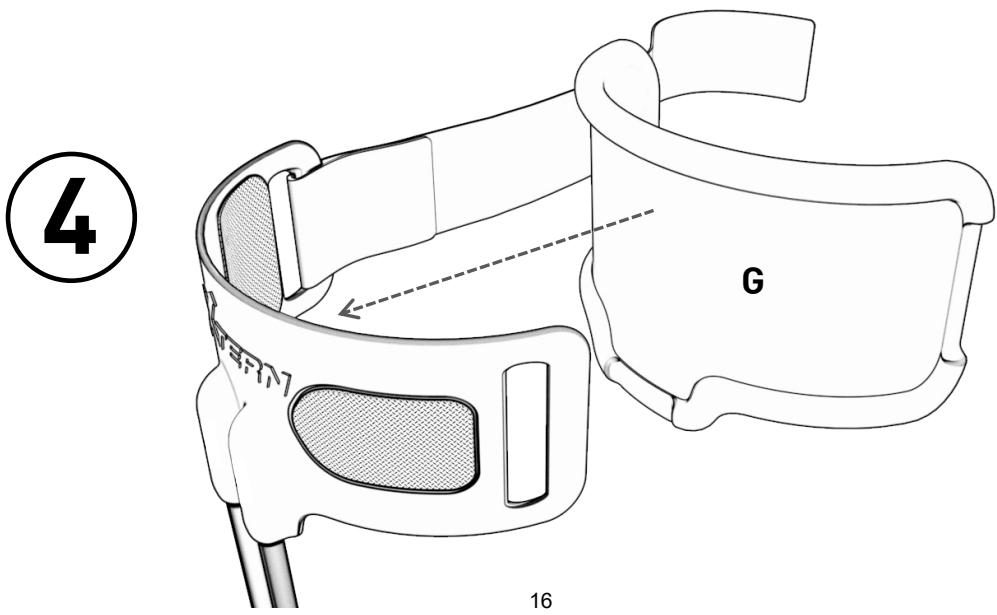
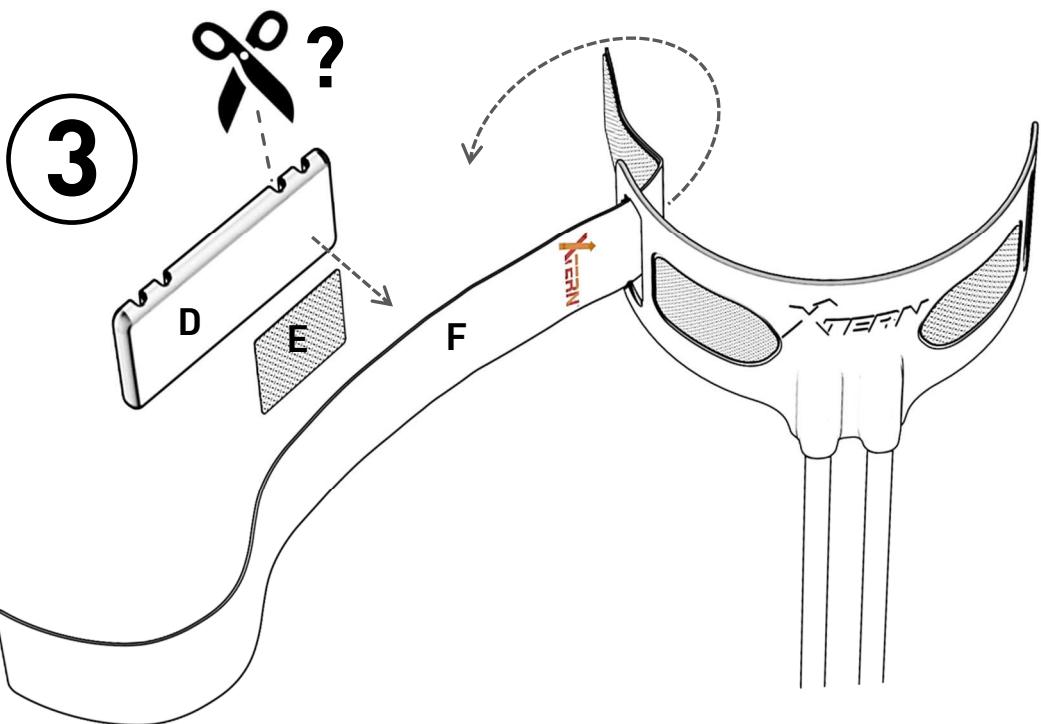
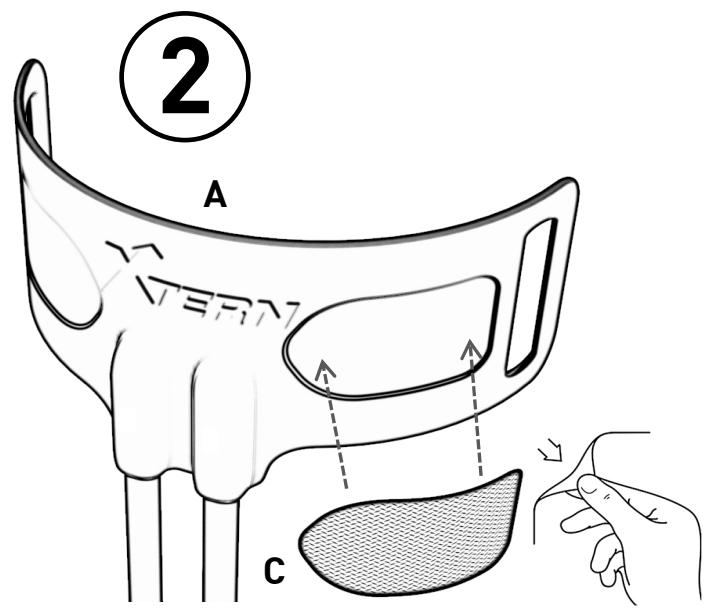
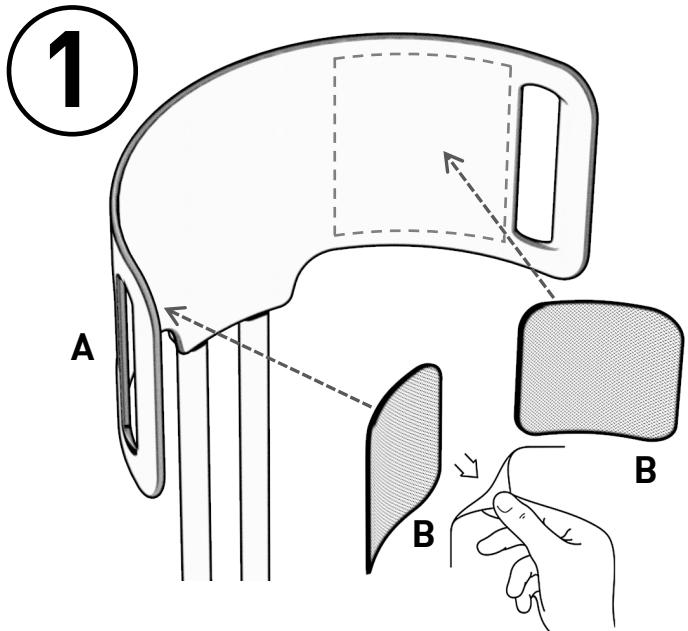
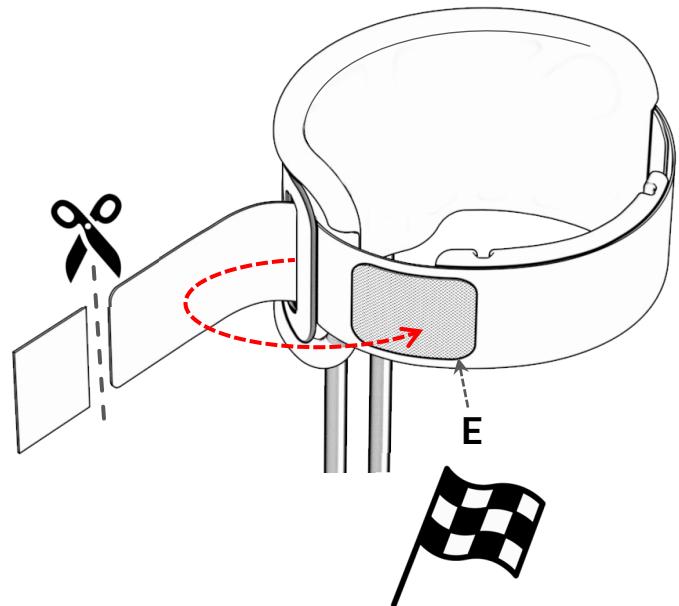
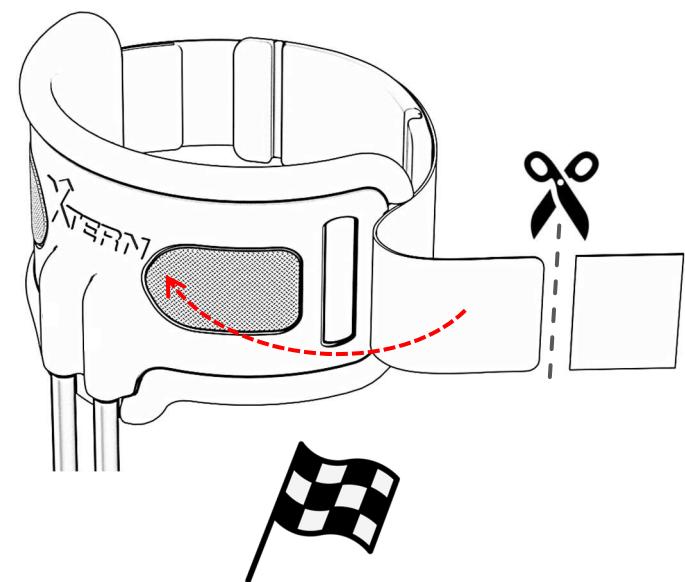
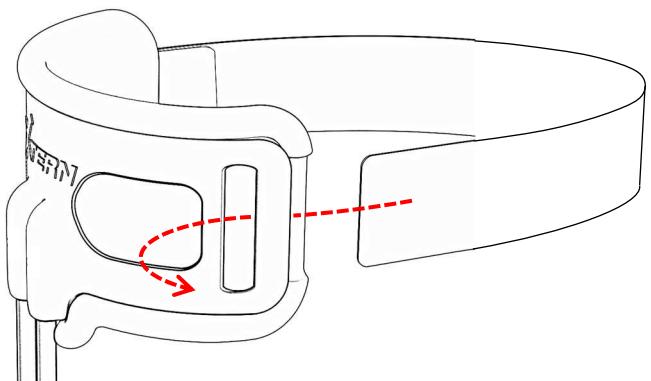
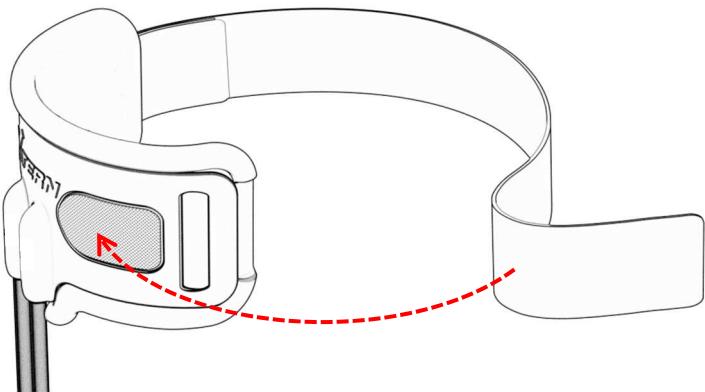
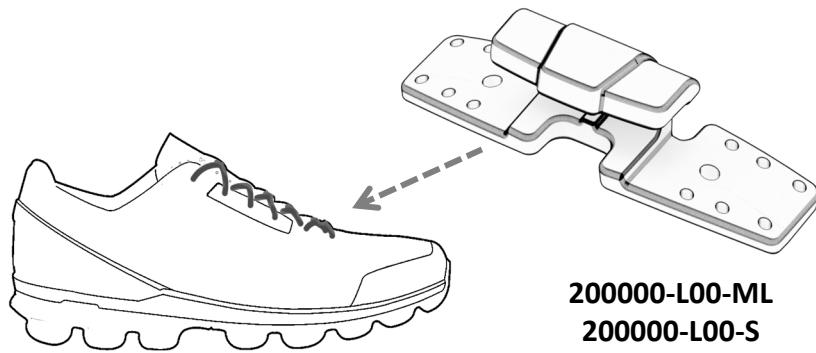
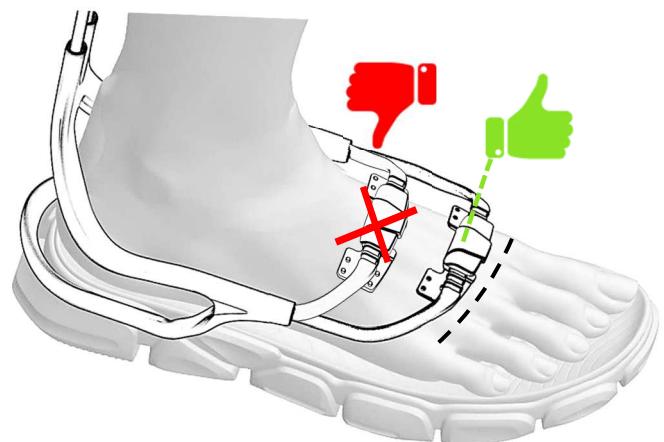
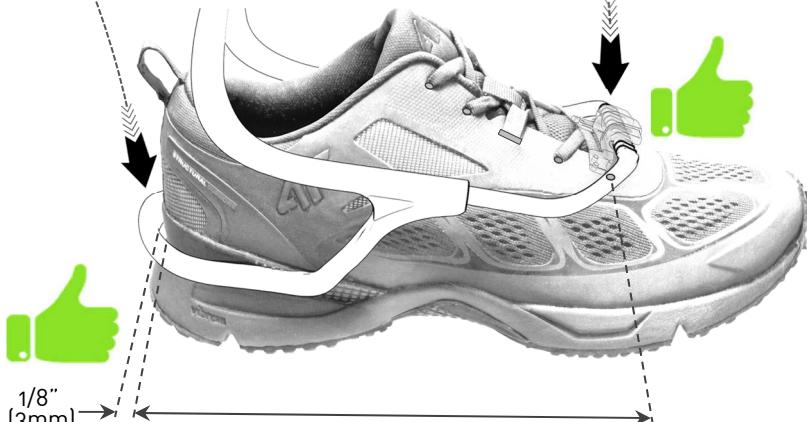
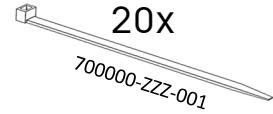
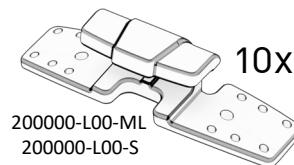
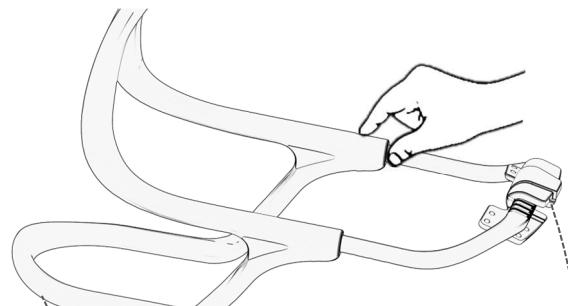
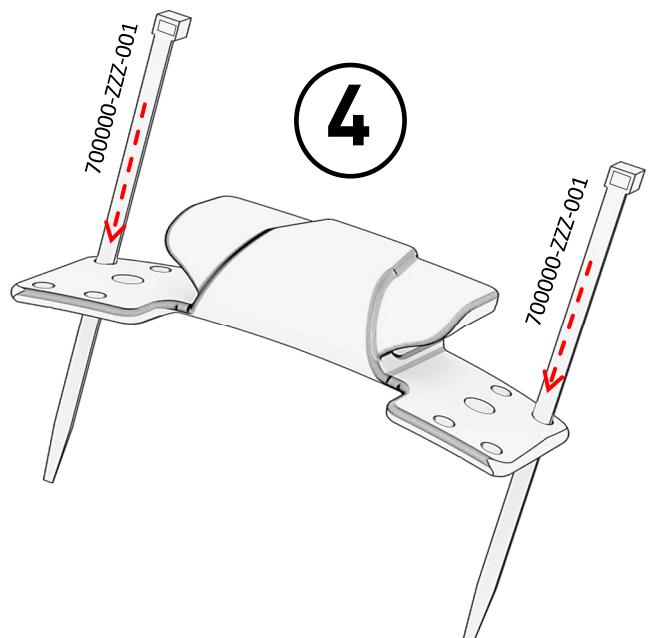
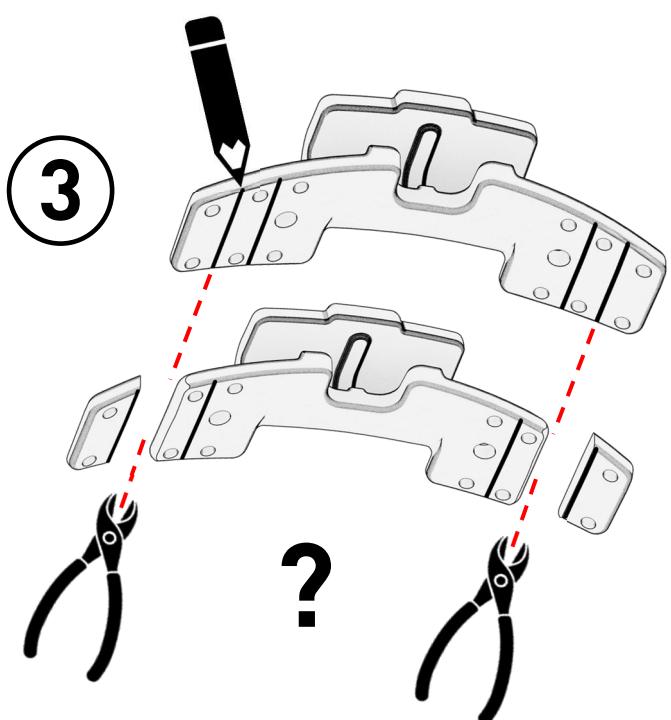
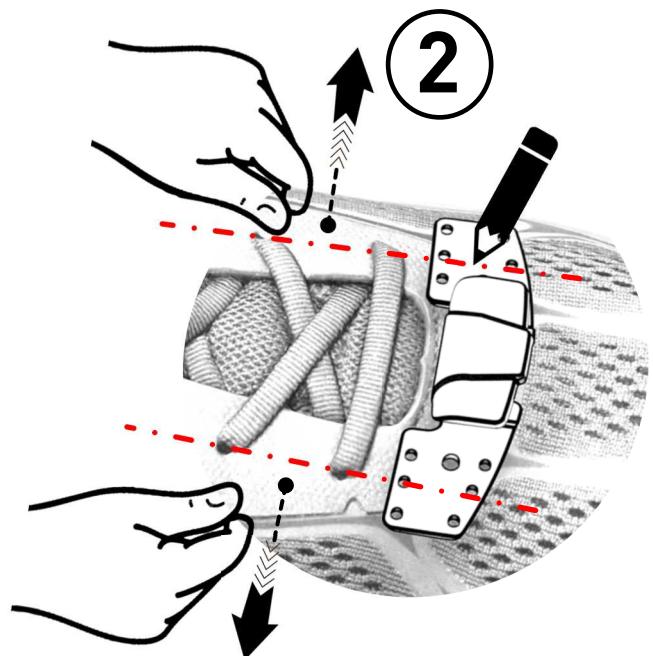
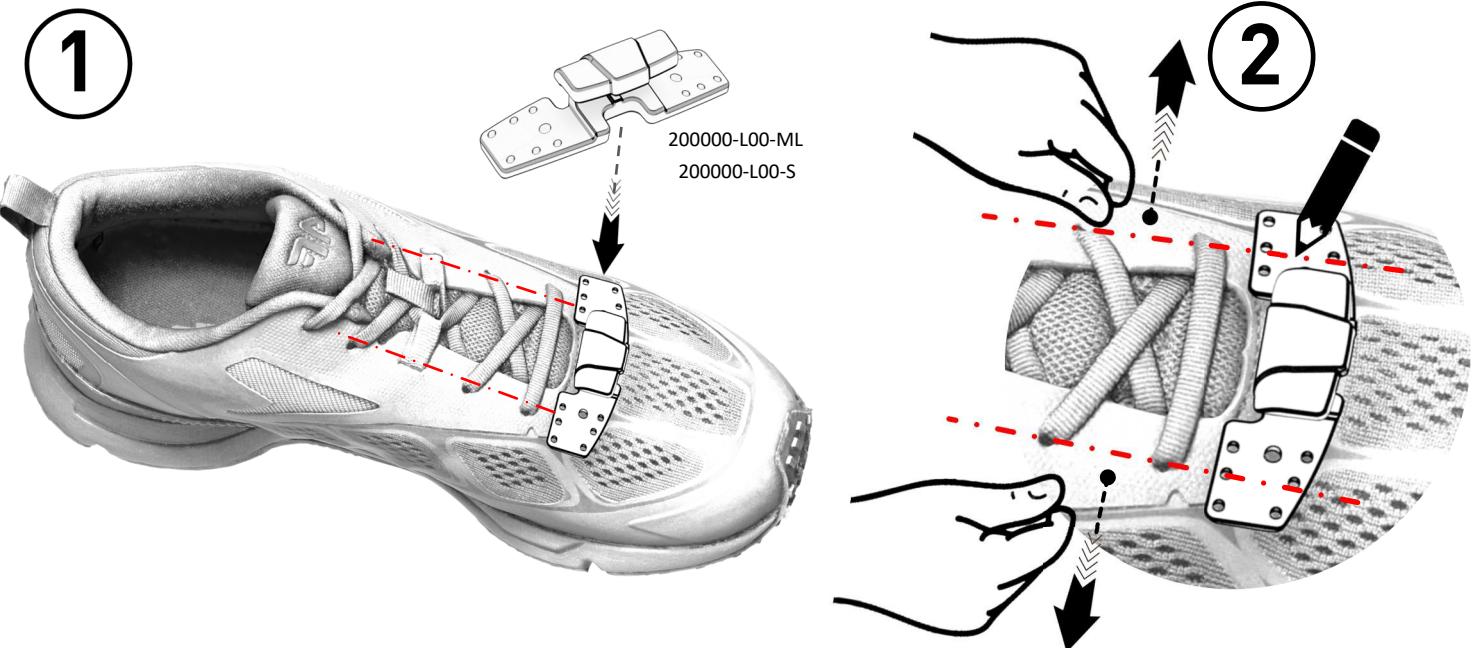
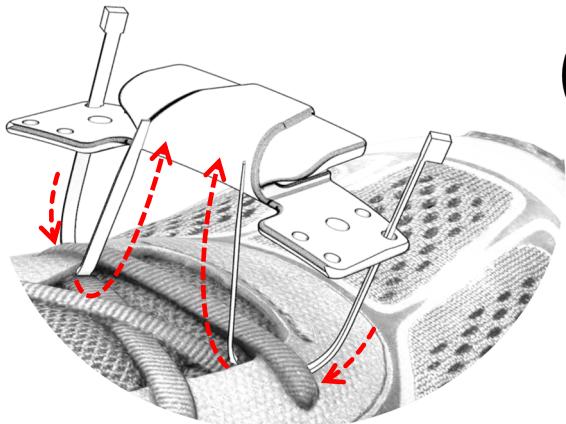
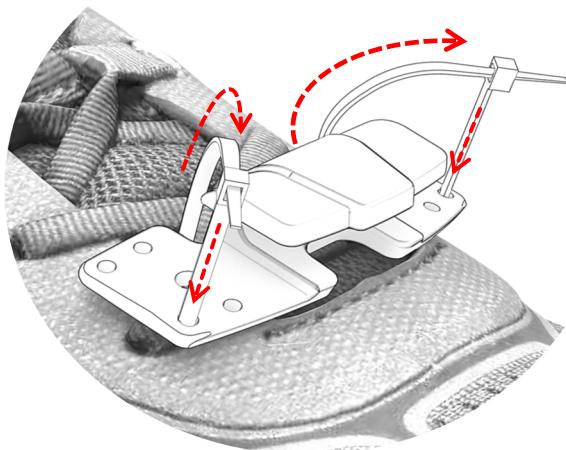
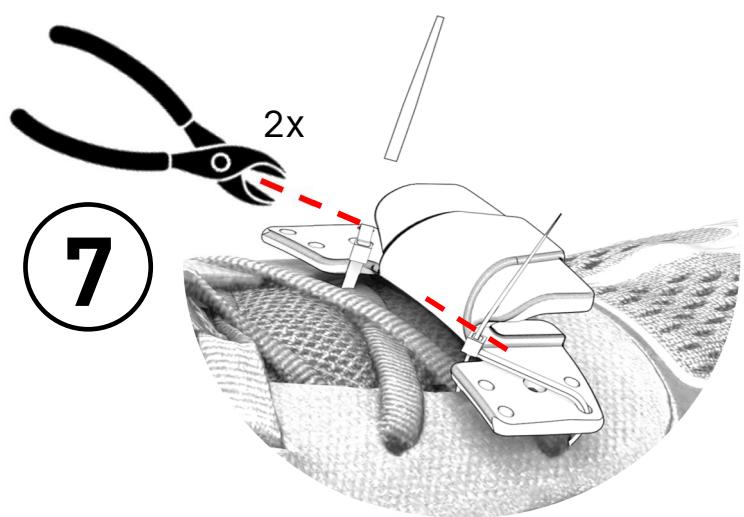
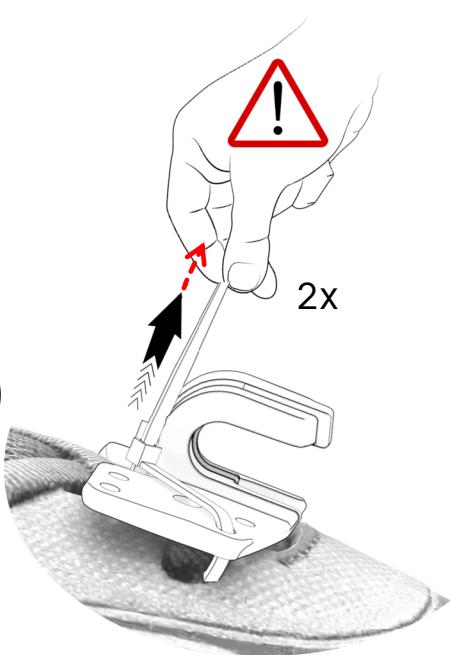
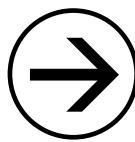
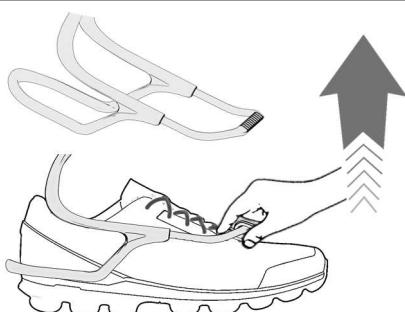
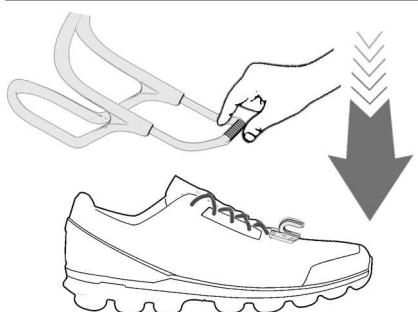
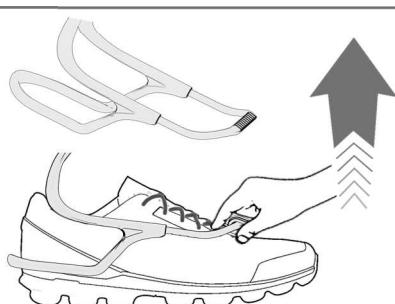
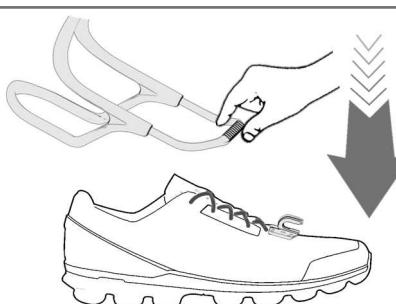
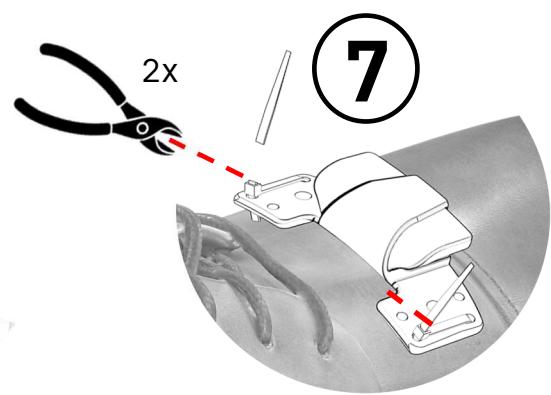
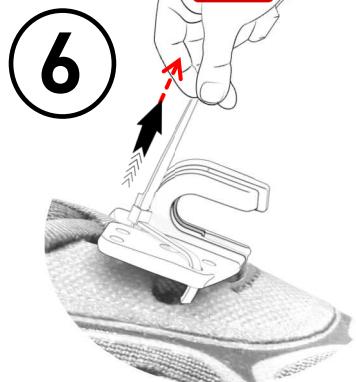
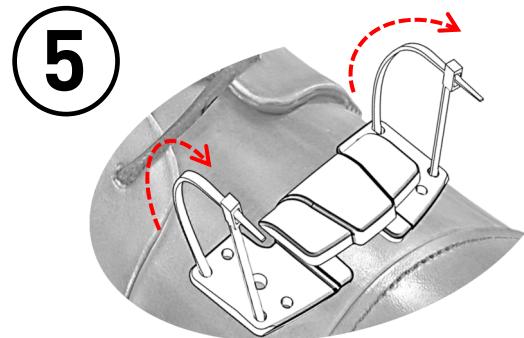
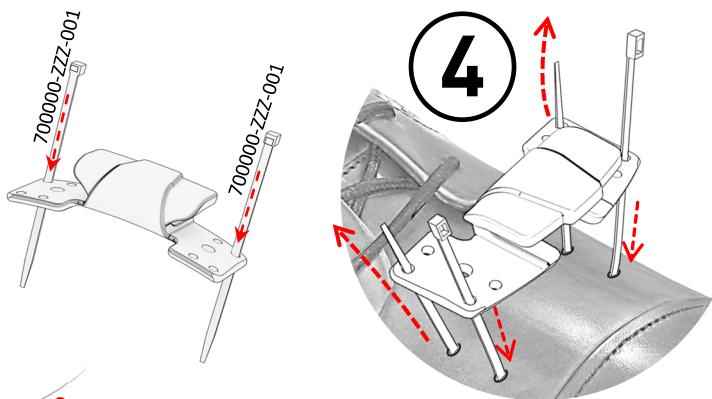
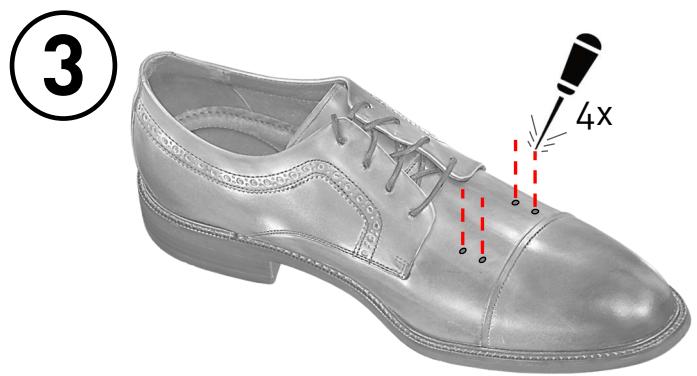
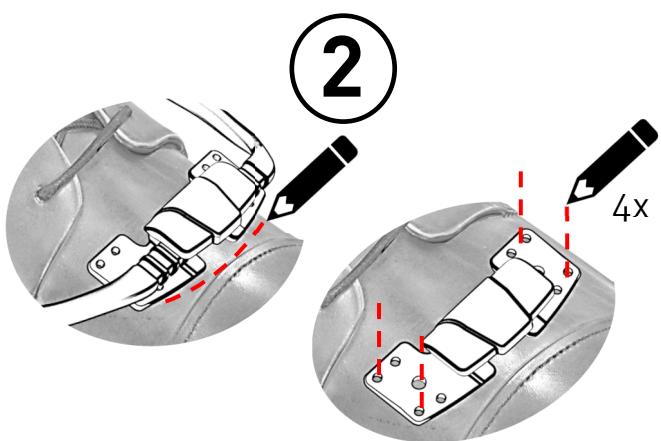
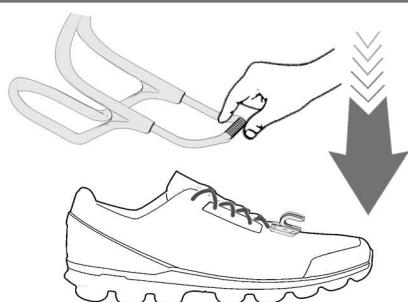
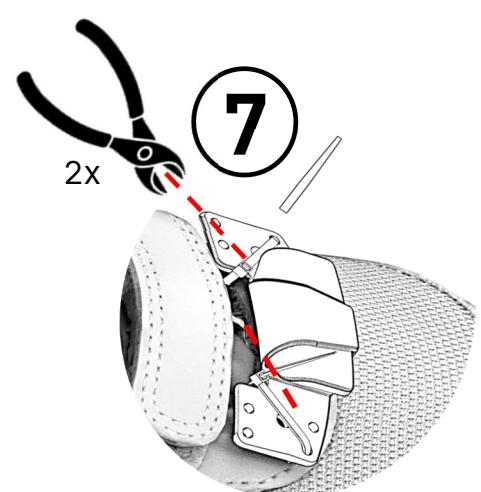
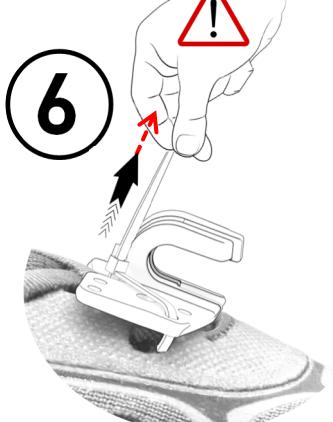
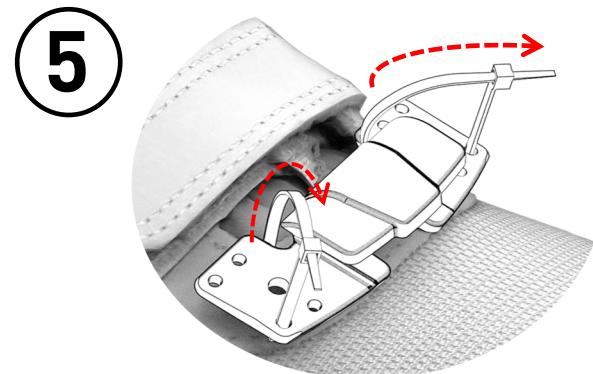
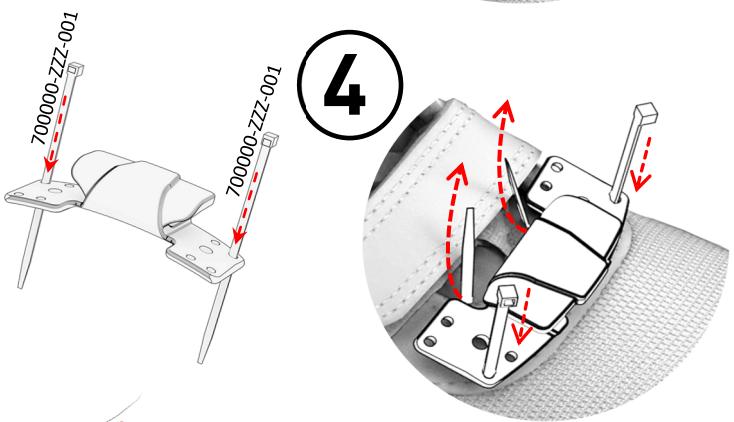
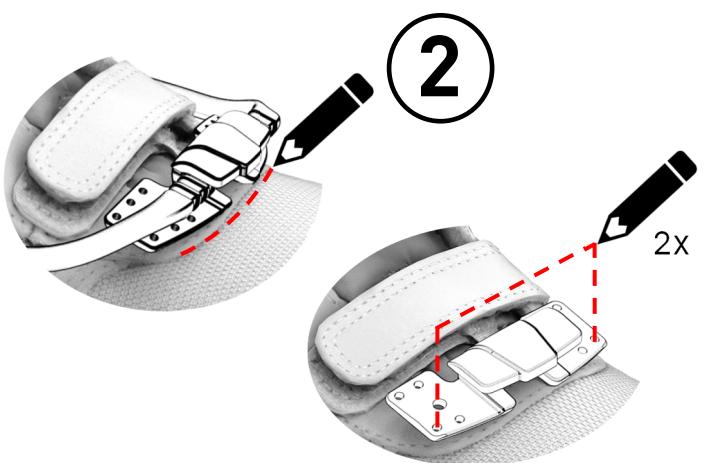
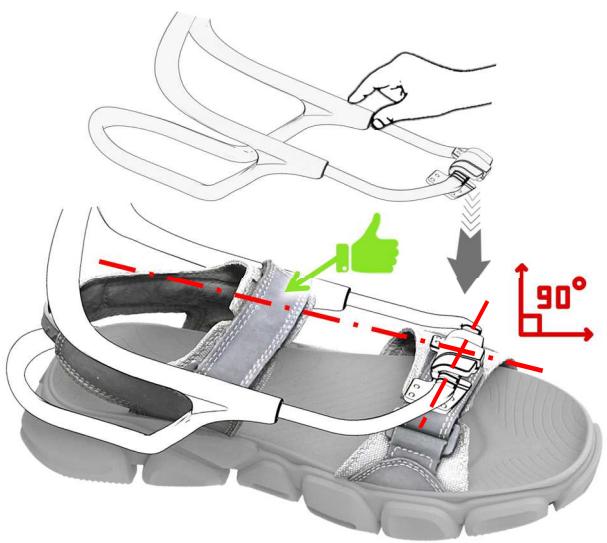
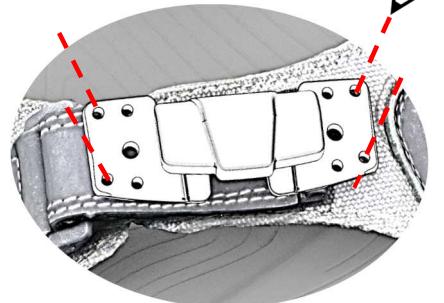
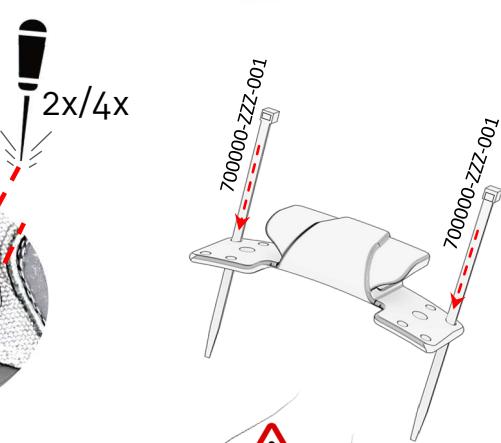
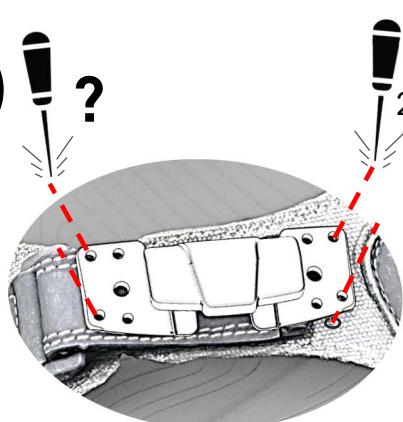
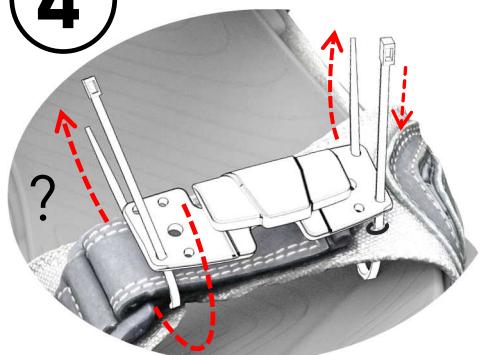
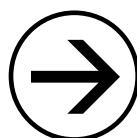
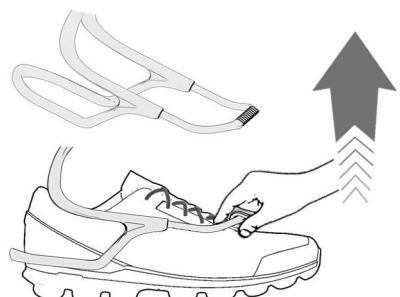
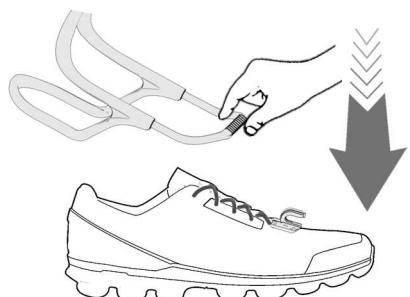
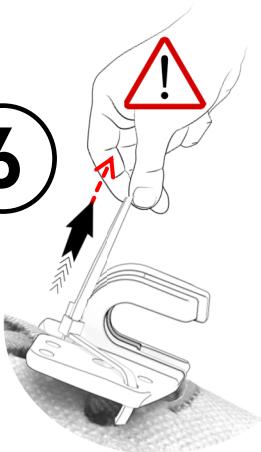
p.24

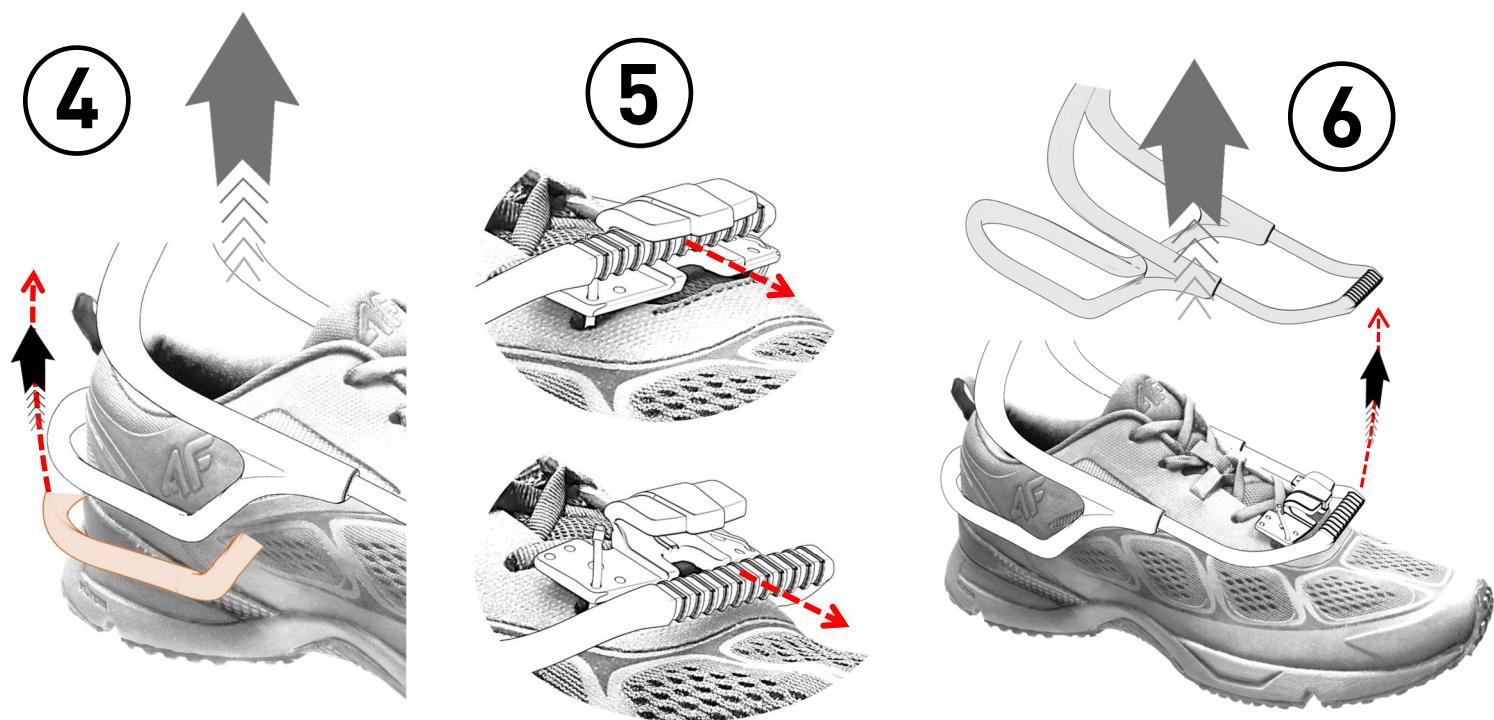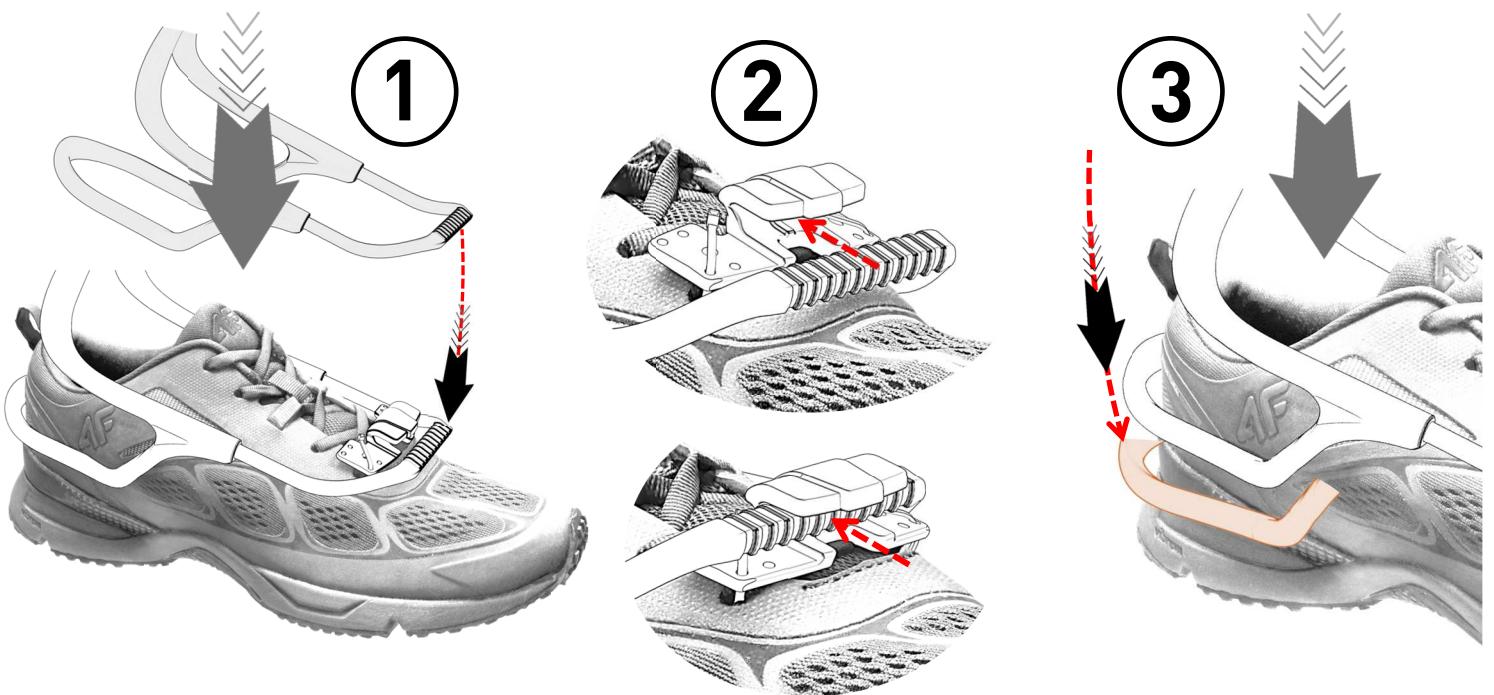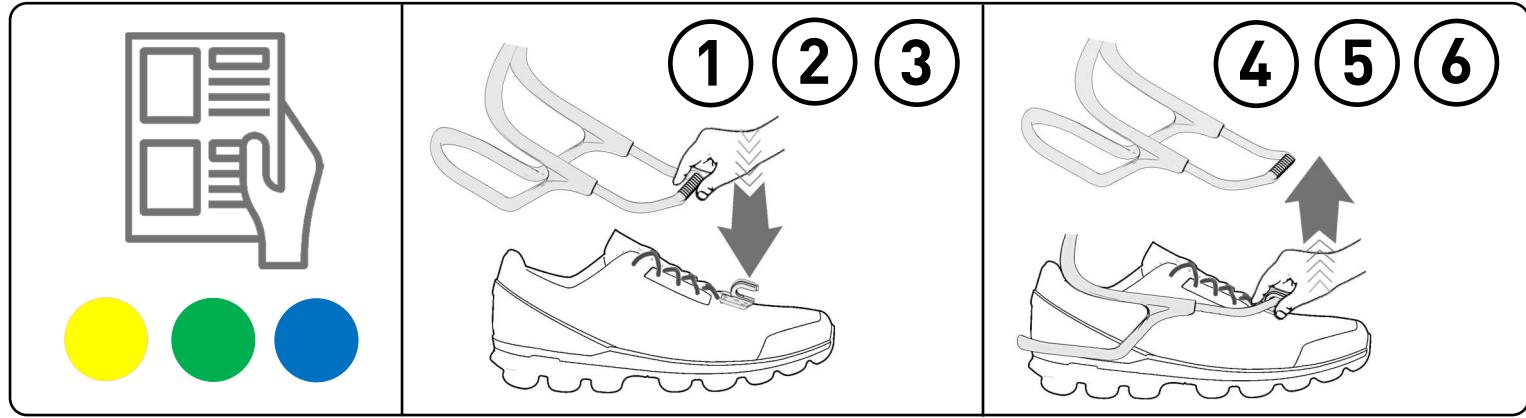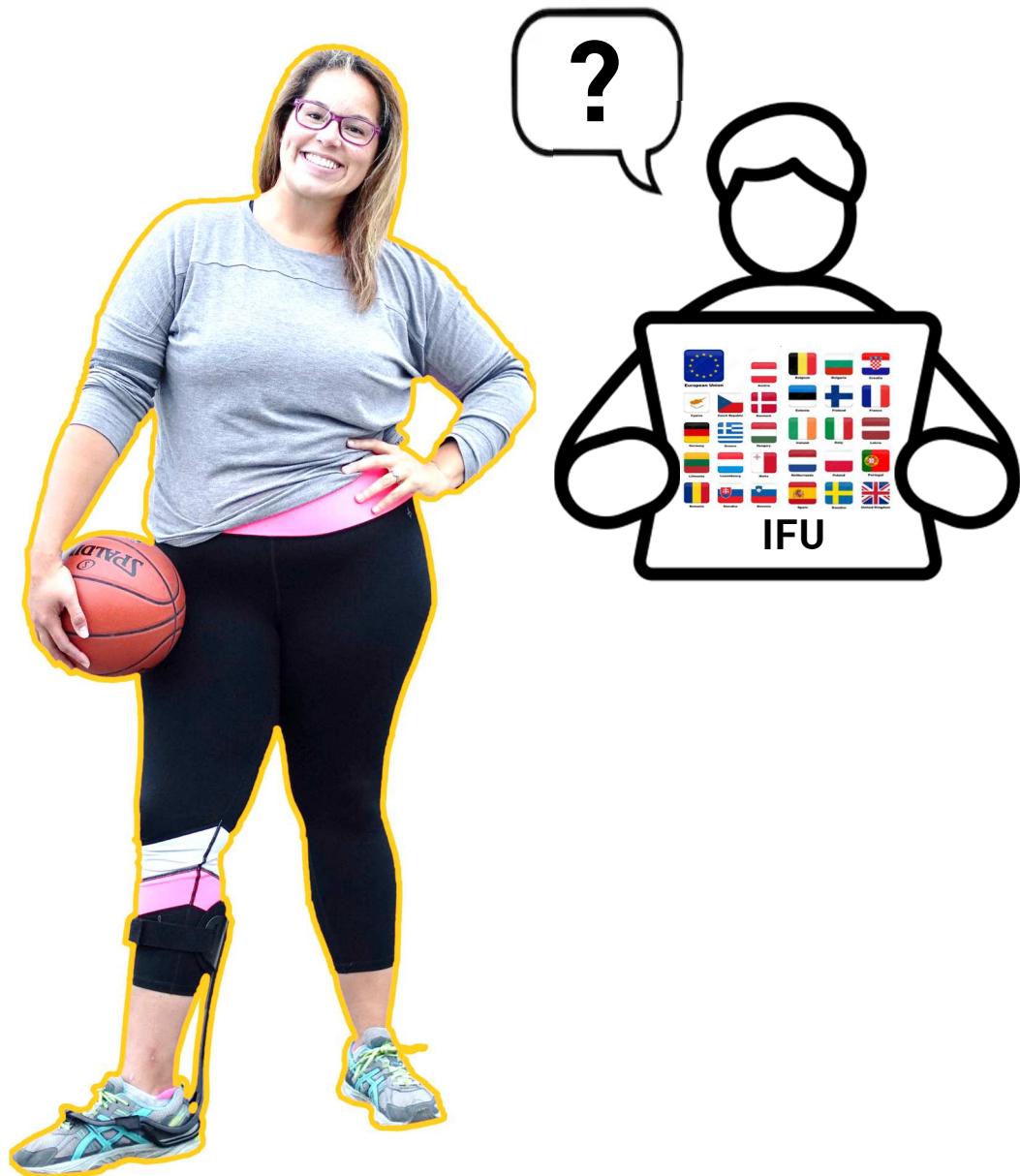1 Introduction

1.1 Copyright and intellectual property

The copyright for these assembly and use instructions is held solely by the manufacturer. Additionally, all rights are reserved, particularly with regard to patent grant or utility model registration. Violations of the above-mentioned provisions will result, where applicable, in the obligation to pay compensation for damage.

Intellectual Property – Patent numbers
U.S. Patent: 6,292,534
CA Patent no.: 2692534
European Patent: EP2533734

1.2 Foreword

We would like to thank you for your confidence in TurboMed and the XTERN SUMMIT. This orthosis was designed by TurboMed Orthotics Inc., located in the province of Quebec, Canada.

This new-generation orthotic solution was developed specifically for people with foot drop.

The XTERN SUMMIT orthosis is designed to meet your needs for performance and comfort.

We very much encourage you to read these instructions and address any questions and comments to your authorized provider before using or assembling your new orthosis. In order to ensure that the equipment works properly and safely, you must strictly follow all of the instructions provided in this document and observe the specified usage restrictions.

If, at any time, you have any concerns about the way your device is working or if you have any questions, do not hesitate to contact your authorized provider.

To contact the customer service department of TurboMed Orthotics, please visit:

www.TurboMedOrthotics.com

1.3 Use

Intended use: This orthosis is intended SOLELY as an orthotic fitting for the lower limb and has been designed only for drop foot.

It is important to use it in accordance with the indications.

1.4 Safety symbols used in this guide

The symbols below are used in this guide to identify the warnings and useful advice that you should observe. It is very important to read and thoroughly understand them before using our product for the first time.

Warning: this symbol means that there is a potential risk. Failure to heed a warning may result in bodily harm or injury.


Notice: recommendations or instructions should be taken seriously.

Product-related caution: concerns a potential material risk. Failure to heed a product caution may result in significant material damage.

2 Presentation of the product

Item	Description
1	MAIN SECTION
2	FRONT HARNESS "U"
3	BACK HARNESS
4	LACE CLIP
5	CALF/SHIN PAD

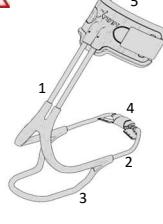3 Indications and contraindications

3.1 Indications

This orthosis is intended for people who have:

-weakness in the muscles that lift the foot, with little to no spasticity (post: back, hip or knee surgery; traumatic injury to the lower limbs; a stroke; traumatic brain injury; multiple sclerosis; Charcot-Marie-Tooth disease; peroneal paralysis; neuromuscular atrophy, etc.).

Minor impairment of the muscles that raise the foot.

Very slight deformations of the foot capable of being corrected with the use of an insole and lateral support.

Indications for this product must always be determined by your doctor or prescribing physician.

3.2 Contraindications

This orthosis is not designed for cases of:

-moderate to severe spasticity of the lower limb;

-leg ulcers in the posterior part of the calf;

-serious balance problems;

-moderate to severe foot deformities;

-certain pathologies that require a visit to a physician to verify whether the orthosis is appropriate for the client or not.

4 The Product

(The product should be used only under the supervision of a qualified physician. MD/DO/DPM)

4.1 Usage restrictions

Do not use this product to prevent contractures (shortening of muscles), etc.

Avoid going near a source of high heat, such as a fire, a heating element, embers, flames, etc.

NEVER use an orthosis that belonged to someone else or an orthosis that you believe has been used.

4.2 Warning and general safety instructions (client)

Information :

-please read this document in its entirety before using the product;

-heed the safety instructions in order to avoid injury or damage to the product;

-keep this document in a safe place.

4.3 Warning and general safety instructions (professional)

Information :

-please read this document in its entirety before using the product;

-heed the safety instructions in order to avoid injury or damage to the product;

-show users how to properly use their product and inform them accurately of the safety instructions;

-keep this document in a safe place.

4.4 Caution

This product is for single patient use only.

Use caution when handling the product.

Check the product regularly to see if it still works properly.

Stop using the orthosis if there are changes in, or loss of, functionality and have it inspected by your authorized health care provider.

Observe the legal and insurance regulations related to operating motor vehicles of any type and have your driving skills tested by the competent authority.

In general, the physician will determine the daily duration of use and the total period of use.

Only certified health care providers are authorized to perform the initial fitting and first application of the product.

Show the patient how to properly handle and care for the product.

Inform the patient how to properly handle and care for the product if unusual changes to the body are observed (ex., the development of a wound, swelling or pain, etc.).

Ensure that the orthosis is applied correctly so that it fits correctly.

Tell the patient to check, prior to each usage, that the product works properly and that it shows no signs of wear or damage.

Direct the patient's attention to the fact that they CANNOT continue to use the product if it or one of its components is worn or damaged (ex., cracks, missing part, warping, inappropriate shape of the orthosis, etc.).

5 Application and removal of the product

5.1 Application

1 Install the lace clip to an appropriate shoe.

2 Open the calf/shin band.

3 Insert the orthosis into the shoe.

4 Put on the shoe and orthosis, pushing back on the orthosis.

5 Close and tie the shoe.

6 Tighten the calf/shin band so as to comfortably secure the top part of the brace to the leg.

Be careful not to over tighten.

5.2 Removal

1 Open the calf/shin band.

2 Untie your shoe.

3 Remove your foot from the shoe.

5.3 Final inspection

Ask the patient to take a few steps to try out the product and to test it on slopes, ramps, while running, and on stairs. Make adjustments, if necessary.

6 Care and cleaning

Care and Cleaning should be done on a regular basis. Clean the rigid parts of the product only with mild soap and water or a damp cloth. The so-called "soft" parts, such as the calf/shin pad, the band, etc., should be cleaned by hand using mild soap, then air-dried. Avoid any source of direct heat (ex., solar radiation, a radiator, fire, etc.).

7 Guarantees

7.1 Guarantee and assistance

The XTERN SUMMIT orthosis was manufactured by TurboMed Orthotics Inc. TurboMed guarantees its orthosis for a period of three years (3 years) from the date of purchase. If a defect occurs during this period, TurboMed will repair it or replace it at its own expense if sufficient proof is provided—namely, a receipt (i.e. Written Proof of Delivery from your authorized provider)—and if there is proof that the date of the service request is within the guarantee period. The use of unauthorized replacement parts will cancel this guarantee. Contact your authorized provider or TurboMed for additional information in this regard.

Excluded from the guarantee are parts not covered by this guarantee.

-damage caused by improper use resulting from excessive use, negligence, fire or any other unusual accident or unauthorized modifications or repairs;

-damage caused by the use of unauthorized replacement parts;

-most common causes of wear, scratches, abrasions, discoloration or the fading of colors;

-the so-called "soft" parts, such as, but not limited to, the calf/shin pad, the calf/shin band, the calf/shin band pad, the plastic fasteners (tie-wraps), the various pieces of VELCRO®, etc.

7.2 Special satisfaction guarantee

We are so confident in the quality of our product that we offer to refund the cost of your orthosis if you are not entirely satisfied (see above) within 2 months of your purchase.

TurboMed, its distributors and their authorized health care providers, undertake jointly and severally to refund in full the amount paid to the client if he/she is not satisfied with the product within two months (2 months) of the date of purchase of the XTERN SUMMIT orthosis. The client and authorized health care providers will be solely responsible for the shipping expenses related to sending the orthosis back to us. However, prior authorization must be requested.

7.3 Legal information

All legal requirements are subject to the national legislation of the country in question and therefore may vary depending on where the product is used.

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